# BSM270/BSM270B

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### **BSM270/BSM270B User Manual for Measurement Guide and Setup**

Thank you for purchasing the BSM270/BSM270B. This user's manual describes all the features of the BSM270/BSM270B. Please read before use and keep it in a safe place. By following the manual instructions, you will be able to use the BSM270/BSM270B more safely and effectively.

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## Please read the warning before using this User Manual.

### WARNING:

Users may be injured or even killed if they do not correctly follow the instructions.

### CAUTION:

Property may be damaged and users may be injured if they do not correctly follow the instructions.

### NOTE

Referring to notes can help improve equipment use.

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---

# InBody

Under international copyright laws, this User Manual may not be copied, translated, or converted into any other media, in whole or in part, without the written consent of InBody Co., Ltd. This User Manual may have typographical errors and the contents are subject to change without prior notice. InBody Co., Ltd. is not responsible for any damage caused by not complying with this User Manual.

For more information, such as the description of more functions and features of the BSM270/BSM270B unit, you can check out and download the product details on the InBody website ([inbody.com](http://inbody.com)). Product appearance and specifications are subject to change to improve performance without prior notice.

If there is a problem with the BSM270/BSM270B unit or if you have any clinical questions, please contact us at:

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# BSM270/BSM270B

## User's Manual Table of Contents

<b>I. Installation</b>	
A. Product configuration	8
B. Installation environment	9
C. How to install	9
D. settings	15
E. Maintenance precautions	18
<b>II. Measurement</b>	
A. Precautions before measurement	19
B. Measurement method	19
C. Measuring posture	21
<b>III. Transport and storage</b>	
A. How to repackage	22
B. Transport and storage environment	25
<b>IV. Troubleshooting</b>	26
<b>V. Other</b>	
A. Safety information and precautions	28
B. Appearance and function	31
C. Indication requirements	33
D. Product specifications	34

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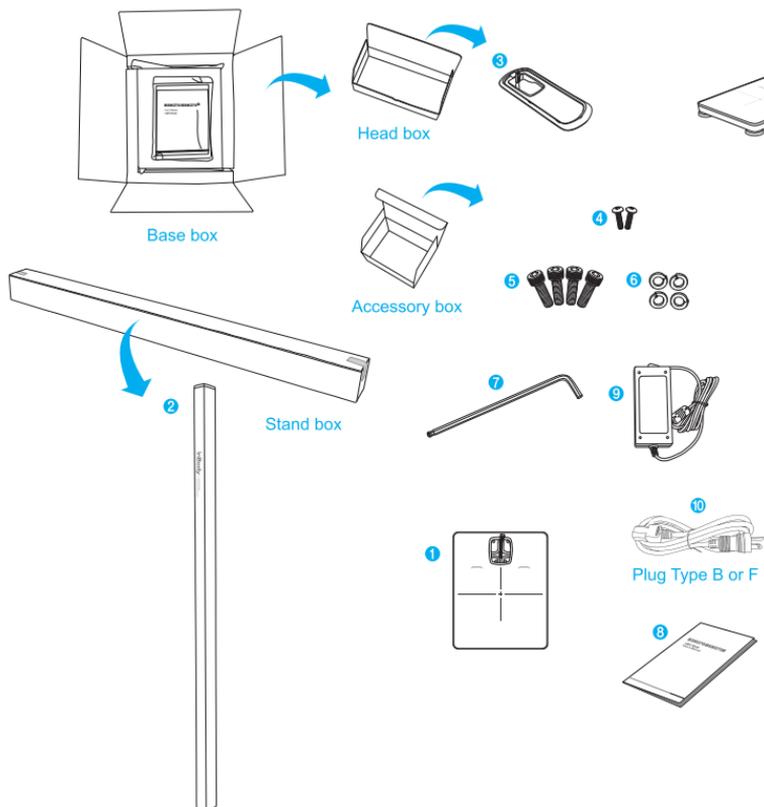
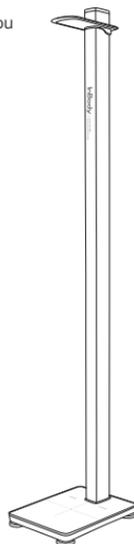
# I. Installation

## A. Product configuration

The product configuration for the BSM270/BSM270B unit is as follows: Make sure you have all of the parts listed.

• Before installation, please make sure that there is no damage to any part.

- 1 Base 1EA
- 2 Stand 1EA
- 3 Head bar 1EA
- 4 Head screws 2EA
- 5 Base screws 4EA
- 6 Base washers 4EA
- 7 Hex key wrench 1 EA
- 8 User's Manual 1 EA
- 9 Adapter (12V 3.0A / 3.33A) 1 EA
- 10 Power cable 110V or 220V(AC250V 10A) 1 EA

## B. Installation environment

Before you install the BSM270/BSM270B unit, check the environment. This equipment is suitable for use indoors. If you are installing outdoors, you must meet the criteria below.

Proper temperature	10 ~ 40°C (50 ~ 104°F)
Relative humidity	30 ~ 75% RH
Proper pressure	70 ~ 106kPa

- Two or more people are required when installing the product.

### WARNING

- Before installing the BSM270/BSM270B unit, make sure that the ceiling height is 2.2 meters or higher. The height of the equipment is 2.17 meters. To ensure that the ceiling and equipment do not contact each other; allow at least 3 cm of additional height.

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## C. How to install

1. After opening the base box, remove the packing material and take out the base, head bar, and the other components.

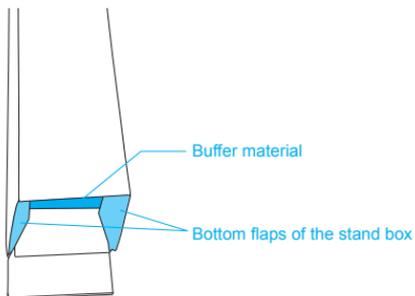
### CAUTION

- If you want to repack your equipment in the future, please keep the packaging provided. Please dispose of all other waste according to local laws and regulations.
- Install the BSM270/BSM270B unit on a flat and vibration-free floor. If installed on an uneven floor, there is a risk of inaccurate measurements or the unit falling.
- Do not wash the BSM270/BSM270B unit with detergent. If liquid detergent flows inside the BSM270 /BSM270B unit, it can cause corrosion and lead to malfunction. To clean the unit, use a soft cloth that is clean and lint-free.

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
2. After opening the stand box, remove the packing material and take out the stand.
3. After removing the unit, put the packing material back into the stand box.
4. Fold in the bottom flaps of the stand box as shown in the figure below.

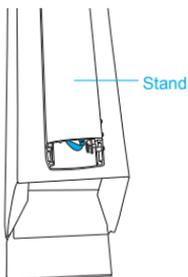5. Place the stand on the stand box so that the protruding screw head is positioned to the right.


 **CAUTION**

- Make sure the cable is properly connected.

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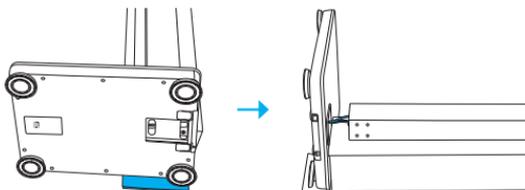
6. Place the base on the finishing part of the stand box and connect the base cable to the stand.

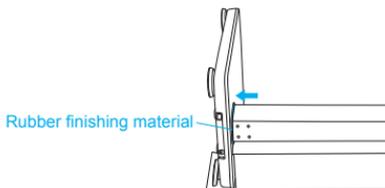7. Hold the base securely and push the stand into the groove of the base.

 **CAUTION**

- If the rubber finishing material is inserted with the stand, the measured value may be inaccurate. Be careful when you insert it.

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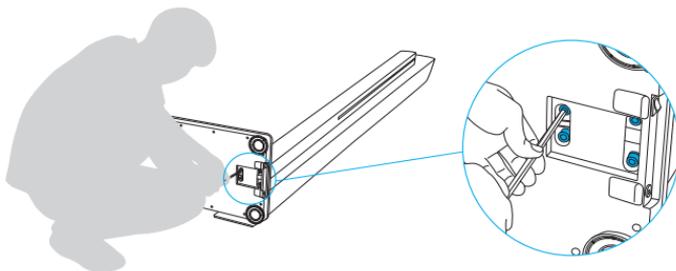
8. Insert the base washer into the base screw.


9. Insert the four screws including the washers into the four screw grooves on the back of the base.

10. Using the hex key wrench, turn the base screw clockwise to secure it tightly.

11. After confirming that the base and the stand are fixed securely, erect the fixed base and the stand to the upright position.

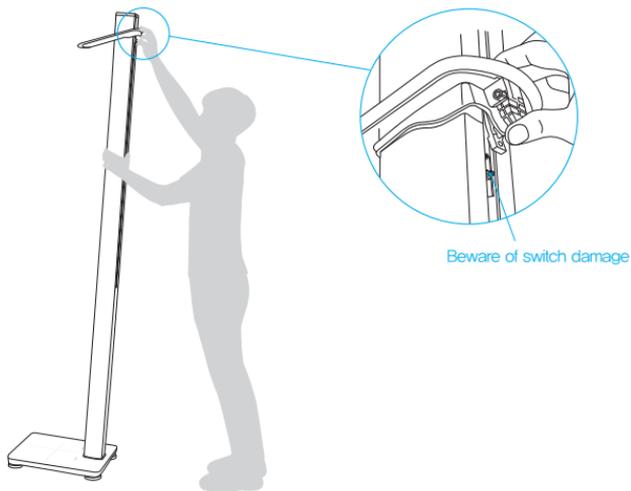12. Tilt the assembled equipment sideways to align the head bar's groove with the stand, and then tilt the head bar as shown in the figure. Insert it up to the screw groove while carefully avoiding damage to the switch.


**CAUTION**

- If the head bar is not tilted during the insertion, the product may be damaged and may not function. Please be careful during this step.
- Do not push the operating part forcefully, as this may cause a malfunction.

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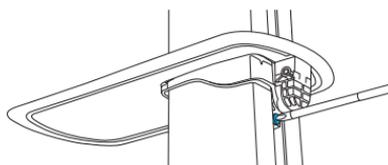13. Align the grooves in the stand screws to match the grooves in the head bar screws, then insert the head screws. Use a Phillips screwdriver to turn the head screws clockwise to secure it tightly.


**CAUTION**

- Using excessive force may damage the screws.

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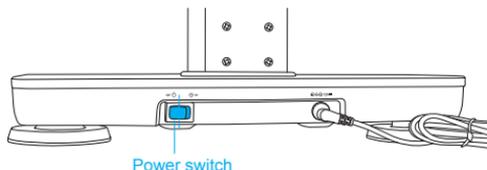14. Connect the adapter and power cord to the power input terminal on the back of the base.

**WARNING**

- Install the equipment where it is easy to unplug the power cord.
- Do not plug or unplug the power cord with wet hands. It may cause electrical shock.
- Use an outlet that is connected to a standard compliant power source (AC 100-240V). Using a different outlet may cause fire or malfunction.
- When using an outlet with multiple terminals, use an outlet or extension cable with sufficient power capacity.
- Do not disassemble the product. Arbitrarily disassembling the operating unit can cause malfunctions, breakdowns, or electrical shock. There are no serviceable parts inside. InBody Co., Ltd. is not responsible for any damage caused by this action.
- Do not allow the BSM270/BSM270B unit to come into contact with other electronic devices while connected to the power source. It may cause electrical shock.

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 **CAUTION**

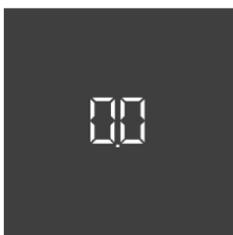
- External device connections are located at the bottom rear part of the machine, which may allow liquids or foreign objects to get inside. Please note that liquids or foreign objects can cause critical damage to the electronic components of the BSM270/BSM270B unit.
- Be sure to only connect the adapter provided by InBody to the power input terminal.
- When connecting the adapter cable to the BSM270/BSM270B unit, insert the adapter cable completely into the power input terminal.
- Be aware that the weight may vary when measuring at an altitude of 2,000 meters or higher.

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15. Turn on the power of the BSM270/BSM270B unit as shown in the image under step 14.

16. After the head operates once, the home screen changes to the screen shown below and to standby mode for measuring.

<When Bluetooth communication is not in use>


<When Bluetooth communication is in use>

## D. Settings
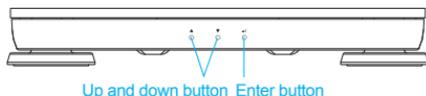
### 1. How to set up

In setup, you can set the measurement mode, offset, unit, head bar position, display orientation, automatic height measurement, display time, weight measurement time, and equipment level check.

- 1) Press the Enter button on the base for more than three seconds, and the setting screen appears as shown in the upper right.
- 2) Use the up and down button (▲, ▼) to move to the item you want to change.
- 3) Press the Enter button to select the item.
- 4) If you have an item that you want to change further, use the direction button to move or select it.  
If no more changes need to be made, press the Enter button for three seconds or more to save the changes automatically and return to the standby screen.

### 2. Set Up Menu

#### 1) (Mode)

The available measurement modes are:

- 1) : Select when measuring both weight and height.
- 2) : Select when measuring weight only.
- 3) : Select when measuring height only.

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#### 2) (Offset)

If you need to adjust the added weight and height values due to clothing or accessories, set the desired offset value to reflect it in the measurement.

- 1) : Adjust the weight offset value. (Adjustment range: +5.0 to -5.0 kg, fixed unit: 0.1 kg / Adjustment range: +10.0 lb to -10.0 lb, fixed unit: 0.2 lb)
- 2) : Adjust the height offset value. (Adjustment range: +50.0 to -50.0 cm, fixed unit: 0.1 cm)

3) **Unit** (Unit)

Select the units used for measurement. You can choose between kg/cm and lb/in.

4) **HPoS** (Head Position)

Set the head bar position. You may choose between **nor** and **HRIF**.

When you change the settings, --- displays and returns to **HPoS** after completion.

① **nor**: Select for heights taller than 1m 60cm (5' 3.0").

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5) **dISP** (Display)

Set the display orientation.

① **nor**: Select if the subject is viewing the display.

② **turn**: Select if the operator is viewing the display.

③ **Auto**: Select if the subject and the operator are both viewing the display together.

 **NOTE**

• The initial factory setting is '**Auto**'.

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6) **HRMo** (Head Mode)

Set whether or not to auto-measure the height.

① **Auto**: Select to automatically measure height after weighing.

② **HRAR**: Select to measure height when the Enter button is pressed after weighing.

7) **dSEC** (Display Sec)

Set how long the display is on.

(Adjustment range: 5-60 seconds)

8) **SPEd** (Speed)

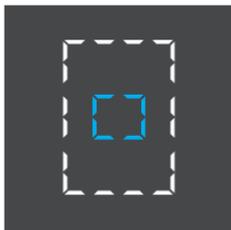
Set the stabilization time for weight measurement.

① **nor**: Select to measure weight when there is no change in movement.

② **FAST**: Select to measure weight after two seconds even if there is a change in movement.

9)  (Level Test)

The current level is represented by a small square position. If it is not level, use the height adjustment base screw located under the base to make it level.


 **NOTE**

- If the small square direction is skewed because it is not level, you can level the machine by raising the screw in the opposite direction.

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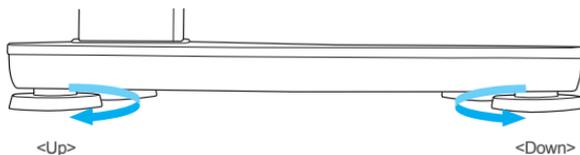 **CAUTION**

- Be careful not to get your fingers caught between the base and the screw.

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## E. Maintenance precautions

### CAUTION

- Do not disassemble the product. Arbitrarily disassembling the operating unit can cause malfunctions, breakdowns, or electrical shock. There are no serviceable parts inside. InBody Co., Ltd. is not responsible for any damage caused by this action.
- For long-term storage, unplug the unit from the outlet.
- Never move the device while the power is on.
- Be careful not to let foreign substances such as food and drinks fall inside the equipment.  
Please note that liquids or foreign objects can cause critical damage to the electronic components of the BSM270/BSM270B unit.
- Do not subject the measuring part (head bar) to strong impact. The measurement may not be accurate.
- Gently wipe the external surface of the machine once a week with a soft cloth that is clean and lint-free.
- Dispose of packaging and other waste according to applicable laws and regulations.
- Never forcefully lower or push the measuring part (head bar) or operating part. It may cause a malfunction.

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### A. Precautions before measurement

#### ⚠ CAUTION

- Be careful not to get your fingers caught between the measuring part and the stand.
- Children or people with reduced mobility can be tested with the help of a manager or assistant.
- If the equipment shakes, use the leveling screw to keep it level.

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### B. Measurement method

To change the measurement mode setting, refer to 'I. Installation, D. Settings, 2. Set Up, 1)  and 6) .

#### 1. Measure weight and height at the same time, and check BMI

When the subject is on the base, the weight appears on the display screen. Once the weight value is stable, the signal sounds and the unit completes the weight measurement. Then, the height is measured.

If it is set to  (automatic measurement)' in the preferences, the height is automatically measured.

If it is set to  (manual measurement)', press the 'Enter' button to start the height measurement.

<When Bluetooth communication is not in use>

<When Bluetooth communication is in use>

#### 2. Measure weight only

When the subject is on the base, the weight appears on the display screen. Once the weight value is stable, the signal sounds and the unit completes the weight measurements.

<When Bluetooth communication is not in use>

<When Bluetooth communication is in use>

### 3. Measure height only

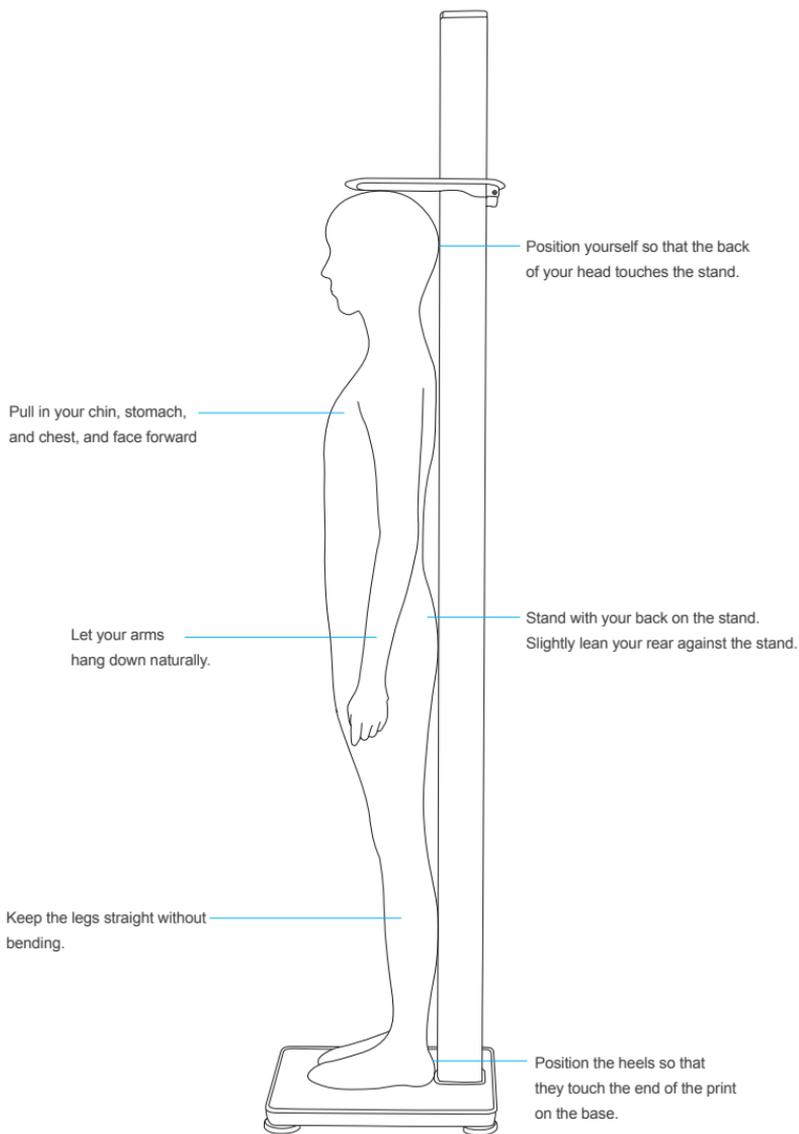
Height measurement methods include 'Auto' (automatic measurement) and 'MANU' (manual measurement). If the Preferences **HTMO** are set to 'Auto' (automatic measurement), the height measurement will start automatically. If set to 'MANU' (manual measurement), press the 'Enter' button to start the height measurement.

- \* The BSM270B unit can be connected to an InBody unit using built-in Bluetooth communication. The connection method can be found in the applicable InBody User's Manual.

### C. Measuring posture

For accurate measurement results, it is necessary to maintain correct posture during measurement.


### III. Transport and storage


#### A. How to repackage

Before repacking the equipment, be sure to turn off the power and remove the adapter. Be careful not to damage the head bar and cables during the repacking process.

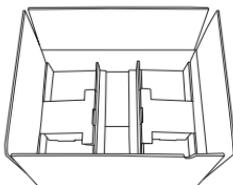1. Prepare the original BSM270/BSM270B base box for repacking.
2. Paste two paper boxes on both ends of the packing box as shown below.

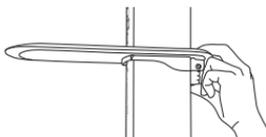3. Be sure to turn off the equipment.
4. Grab the back of the head bar and slowly lower it by hand.


#### WARNING

- If you hold the front of the head bar and lower it, there is a risk of damage to the unit.
- Lowering the head bar quickly may risk damage to the unit.
- Never apply force to the head bar unless it is a special case, such as repackaging. There is a risk of damage.

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
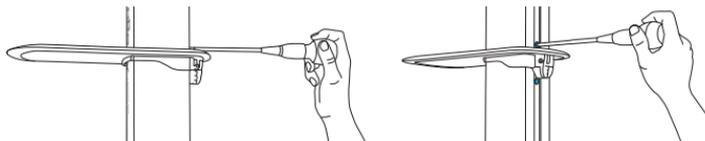
5. Remove the head bar by loosening the head screw with a screwdriver.

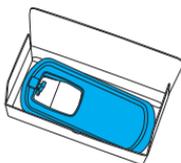
6. Insert the head bar into the head box.

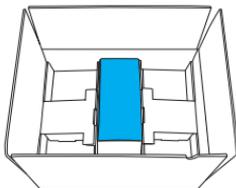7. Put the head box in the base packing box.

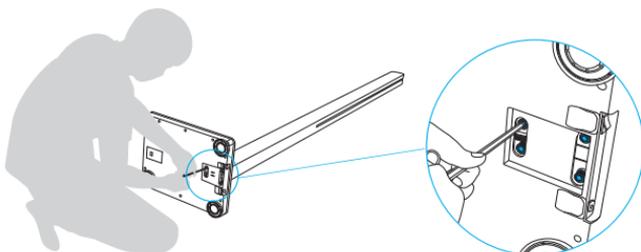
8. Lay the equipment on its side to separate the stand and base.


9. Insert the adapter, cable, and screws into the accessory box.

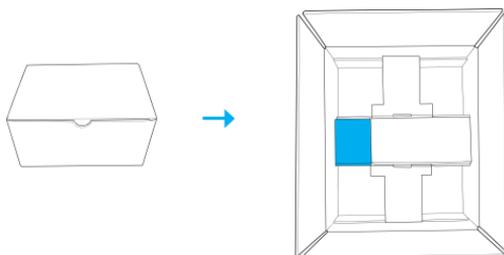
10. Put the accessory box in the base box.

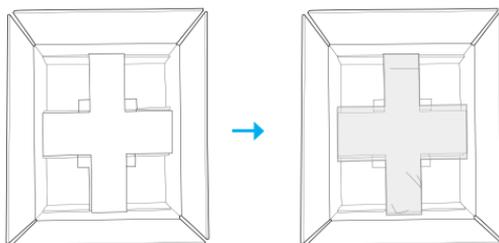
11. Raise the arrow pad and put it in the foam bag.

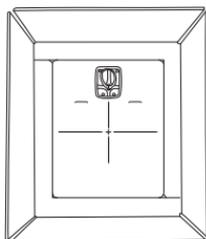
12. Position the polyethylene foam pad and place the separated base as shown below.

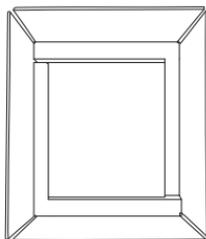13. Top with the foam bag and put the box pad on top of the bag.

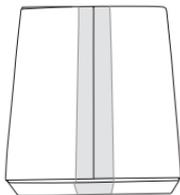
14. Close the packing box and seal it with tape.


15. Insert a polyethylene foam pad between both ends of the stand.

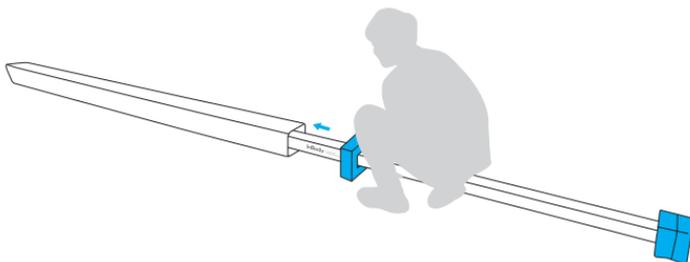16. Push the stand into the stand box.

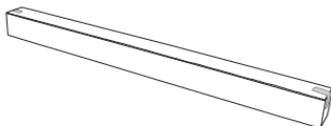17. Close the packing box and seal it with tape.

## B. Transport and storage environment

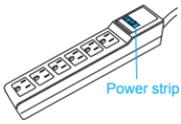
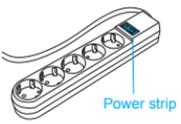
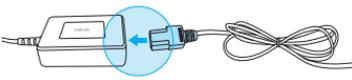
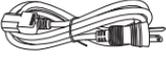
The following environment must be met to safely transport or store the BSM270/BSM270B unit.

Proper temperature	-10 ~ 70°C (14 ~ 158°F)
Relative humidity	10 ~ 80% RH (No Condensation)
Proper pressure	50 ~ 106kPa

## IV. Troubleshooting

Below, you will find troubleshooting information about the BSM270/BSM270B.

If you have problems using your BSM270/BSM270B unit or have questions about your height measurements, first check the troubleshooting information below. If you still have questions after checking the following information, please contact InBody Customer Support.

Issue	Potential Solution
<ul style="list-style-type: none"><li>• There is no power.</li></ul>	<ul style="list-style-type: none"><li>• Plug the power cable completely into the 3-terminal outlet.<ul style="list-style-type: none"><li>&lt;Plug Type B&gt; </li><li>&lt;Plug Type F&gt; </li></ul></li></ul>
	<ul style="list-style-type: none"><li>• When using a power strip, the power will not turn on if the power strip itself is turned off. Check that the power strip is connected to the power cable.<ul style="list-style-type: none"><li>&lt;Plug Type B&gt; </li><li>&lt;Plug Type F&gt; </li></ul></li></ul>
	<ul style="list-style-type: none"><li>• Plug the power cable completely into the adapter terminal. </li></ul>
	<ul style="list-style-type: none"><li>• This may occur when using an adapter not provided by InBody. Be sure to only use the power adapter (DC 12V 3.0A / 3.33A) provided by InBody Co., Ltd.<ul style="list-style-type: none"><li> Power adapter</li><li>&lt;Plug Type B&gt; </li><li>&lt;Plug Type F&gt; </li></ul></li></ul>

Issue	Potential Solution
<ul style="list-style-type: none"> <li>The measured height value is abnormal.</li> </ul>	<ul style="list-style-type: none"> <li>Check if the equipment is installed on a flat and vibration-free floor.</li> <li>The measuring posture must be as instructed. For detailed information on the measurement posture, refer to 'II. Height Measurement, C. Measurement Posture, of this User's Manual.</li> <li>Turn the device off and on.</li> </ul>
<ul style="list-style-type: none"> <li>The weight value is very different from normal.</li> </ul>	<ul style="list-style-type: none"> <li>This occurs when device warm-up is abnormal. If any object is placed on the base, the zero adjustments will not be performed properly and the weight will not be measured accurately. Therefore, turn off the power and turn it on again without anything placed on the base.</li> <li>This occurs when a foreign object is stuck between the base and the base screw. If so, the weight will not be measured accurately. Therefore, remove any foreign objects before use.</li> <li>This occurs when the unit is not level. If it is not level, the weight will not be measured accurately. Please refer to 'I. Installation, D. Settings, 2) Set Up, 9)  ' of this User's Manual for leveling information.</li> </ul>

## V. Other

- The BSM270/BSM270B unit has been designed, manufactured, and inspected by InBody's Quality Assurance system. InBody Co., Ltd. and conforms to ISO9001.

### A. Safety information and precautions

#### WARNING

1. Make sure that children or people with reduced mobility do not attempt to measure alone. However, they can measure with the help of a manager or assistant.
2. Do not insert or remove the power cord with wet hands. It may cause electrical shock.
3. Do not allow children to play near the device. Serious injuries can occur if the equipment falls.
4. To prevent device malfunctions and electrical fires, use an outlet with a proper power source (100-240 VAC).
5. When using an outlet with multiple terminals, use an outlet or extension cable with sufficient power capacity.
6. Do not allow the BSM270/BSM270B unit to come into contact with other electronic devices while connected to the power source. It may cause electrical shock.
7. Do not disassemble the product. Arbitrarily disassembling the operating unit can cause malfunctions, breakdowns, or electrical shock. There are no serviceable parts inside. InBody Co., Ltd. is not responsible for any damage caused by this action.

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## CAUTION

1. Washing  
Unplug the unit before attempting to clean it. Wipe the unit with a soft cloth that is clean and lint-free. Do not use water, benzene, thinner, or alcohol. Spraying liquid on the product may cause malfunction and electrical shock.
2. Peripheral devices  
Remove electromagnetic interference from peripheral devices. This may cause imprecise measurements or malfunctions.
3. Installation  
Install the BSM270/BSM270B unit on a flat and vibration-free floor. If installed on an uneven floor, there is a risk of inaccurate measurements or the unit falling.
4. Use the adapter model provided by the manufacturer. The use of other adapters can cause the device to malfunction.
5. Be careful not to let foreign substances such as food and drinks fall inside the equipment. Please note that liquids or foreign objects can cause critical damage to the electronic components of the BSM270/BSM270B unit.
6. Do not subject the measuring part (head bar) to strong impact or movement during Please delete measurement. The measurement may not be accurate.
7. Do not bend the head bar forcefully, as it can cause damage to the device.
8. To level the product, when using the height adjustment base screw on the bottom of the product, be careful that your fingers do not get caught between the product and the floor. This could cause injury.
9. Dispose of packaging and other trash according to applicable laws and regulations.

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## NOTE

1. If the device is not installed as instructed in this User's Manual, or is affected by interference from peripheral devices, it may malfunction, or the measurement value may not be accurate. Address interference in the following ways:
  - Disconnect the peripheral device that is causing interference from power.
  - Move it physically away. Connect to power sources not used by peripheral devices.
  - If the problem is not resolved after taking the above steps, please contact InBody Co., Ltd.
2. As excessively high or low temperature, humidity, and pressure may affect the operation of the equipment and cause malfunction, use within the range specified in the product specifications.
3. When moving the BSM270/BSM270B unit, be sure to use the packaging material provided at the time of purchase, and do not subject it to severe physical shocks during installation, disassembly, or use.
4. Do not use the unit for purposes other than the measurements of height and weight.
5. Repair and inspection can only be performed by InBody's A/S staff. For repairs and inspections, please contact InBody Co., Ltd.

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## B. Appearance and function

The name and function of each part of the BSM270/BSM270B unit are as follows.

\* Before installation, please make sure that there is no damage to any part.

### 1. The head bar

- 1 Head bar: Used to measure height.

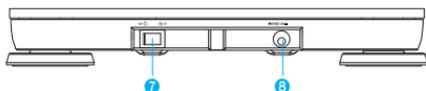
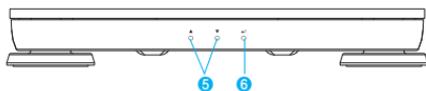
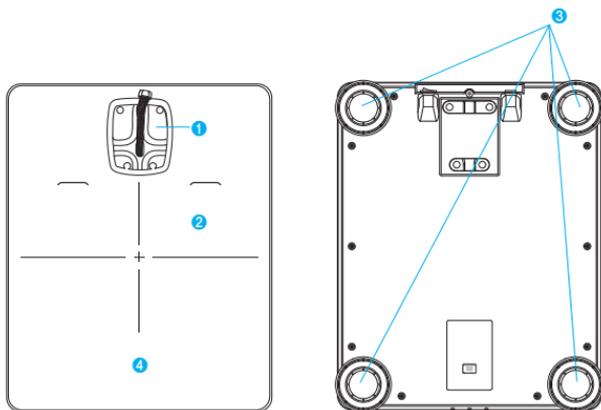
### 2. Stand

- 1 Stand: Provides support to the equipment.

### 3. Base

- ❶ Stand insert: Where the stand is inserted and assembled to the base.
- ❷ Base: What the person stands on when measuring height.
- ❸ Base adjustment screws: Used to level the machine.
- ❹ LCD screen: Displays height and weight measurements on the screen.
- ❺ ▲, ▼ Direction buttons: Used to move toward the desired direction when setting the preferences.
- ❻ Enter button: Used to select or change items when setting the preferences.
- ❼ Power switch: Used to turn the unit on or off.
- ❽ Power input terminal: Connects the power adapter.

## C. Indication requirements

### Safety symbols

WARNING / AVERTISSEMENT, CAUTION / ATTENTION

NOTE / REMARQUE

---

### Other symbols

Manufacturer

European Conformity

Manufacturing number

DC power

---

## D. Product specifications

Product purpose	A device that can automatically measure height and weight	
Measurement items	Height, Weight, BMI	
Measured height	900~2000mm	
Height measurement error range	± 1mm	
Measured weight	10~200kg (22~440lb)	
Outcome indication	Measurement outcome is displayed on the LCD screen	
Zero-point adjustment	Zero adjustments are performed during the warm-up time of 3~15 seconds for weight measurements	
Measurement mode	WH (measures both weight and height) W (measures only weight) H (measures only height)	
Height measurement	AUTO (automatic height measurement after weight measurement)	
	MANUAL (after weight measurement, press the "ENTER" button to measure height)	
Height offset	Can be adjusted between + 50.0cm and -50.0cm in 0.1cm increments	
Weight offset	Can be adjusted between +5.0kg and -5.0kg in 0.1kg increments	
	Can be adjusted between +10.0lb and -10.0lb in 0.2lb increments	
Measurement units	Select between 'kg/cm' and 'lb/in'	
Adapter	Power input	100 - 240V, 50/60Hz, 1.2A
	Power output	DC 12V, 3.33A
External interface	Bluetooth * Applies only to the BSM270B unit	
Equipment size	355(W) × 455(L) × 2170(H); mm	
Equipment weight	14kg(30lb)	
Operating environment	Temperature 10 ~ 40°C(50 ~ 104°F), humidity 30 ~ 75% RH, air pressure 70 ~ 106 kPa	
Transport and storage environment	Temperature -10 ~ 70°C(14 ~ 158°F), humidity 10 ~ 80% RH, air pressure 50 ~ 106 kPa (No condensation)	

- \* The above specifications are subject to change without notice to improve the unit's appearance and product performance.

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