LookinBody 120

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
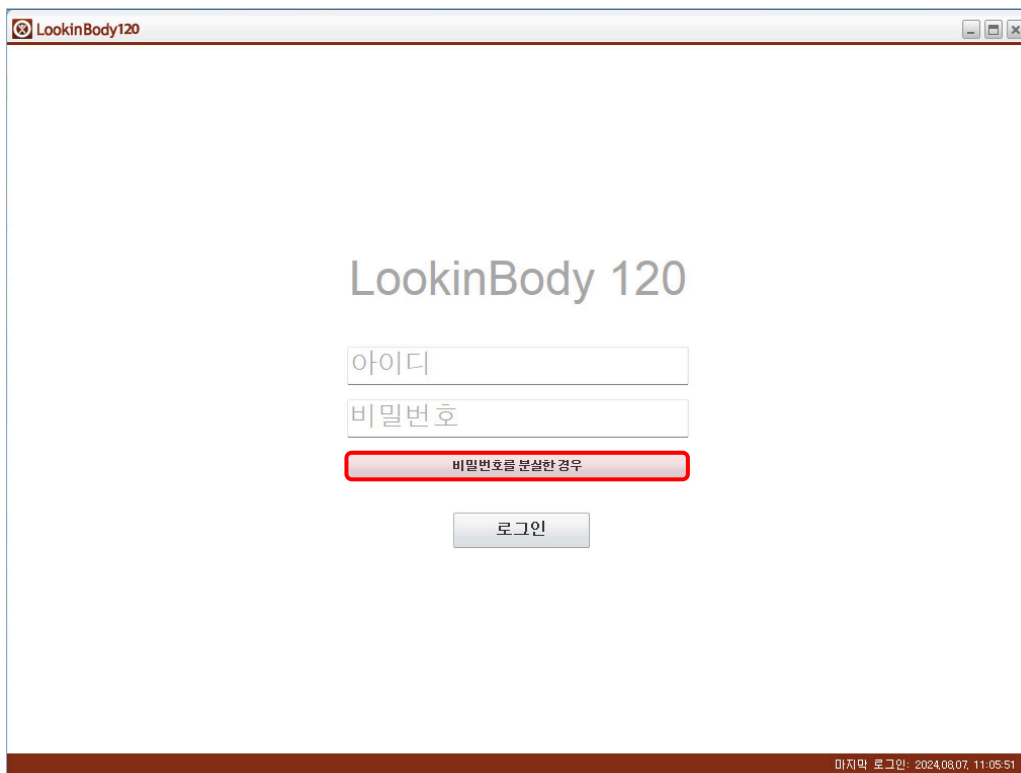
The screenshot shows a web browser window titled "LookinBody120". The main content area displays the "LookinBody 120" logo at the top. Below the logo are two input fields: "아이디" (ID) and "비밀번호" (Password). A button labeled "비밀번호를 분실한 경우" (Forgot Password) is highlighted with a red border. Below this button is a "로그인" (Login) button. At the bottom right of the browser window, there is a timestamp: "마지막 로그인: 2024.03.07, 11:05:51".

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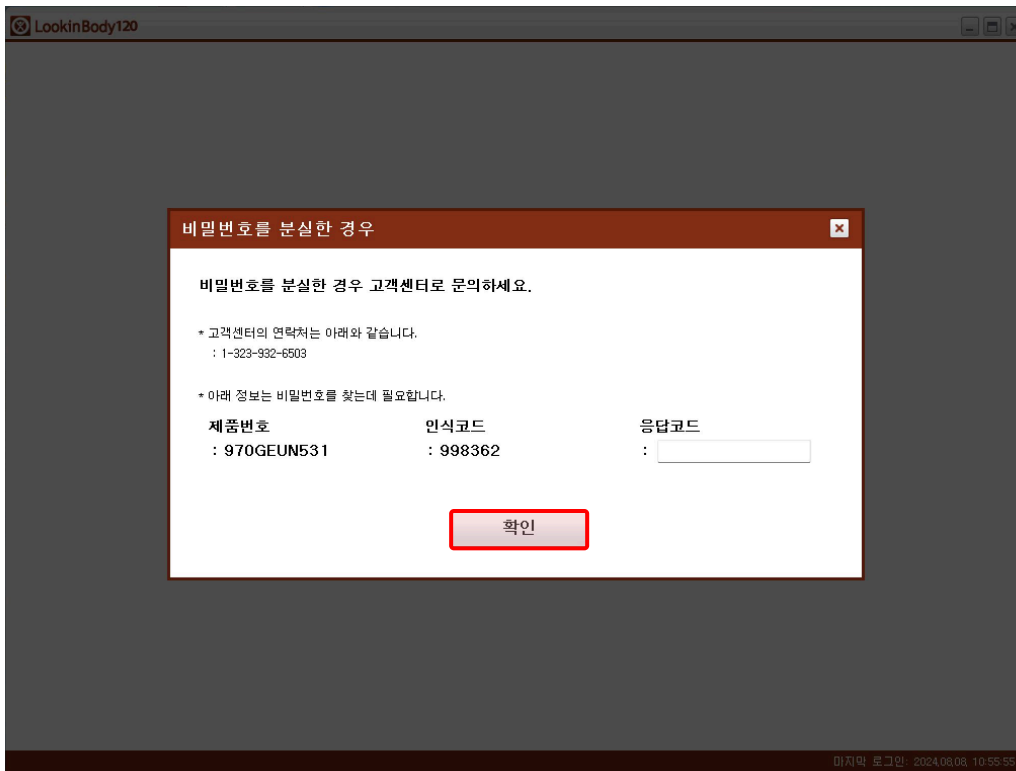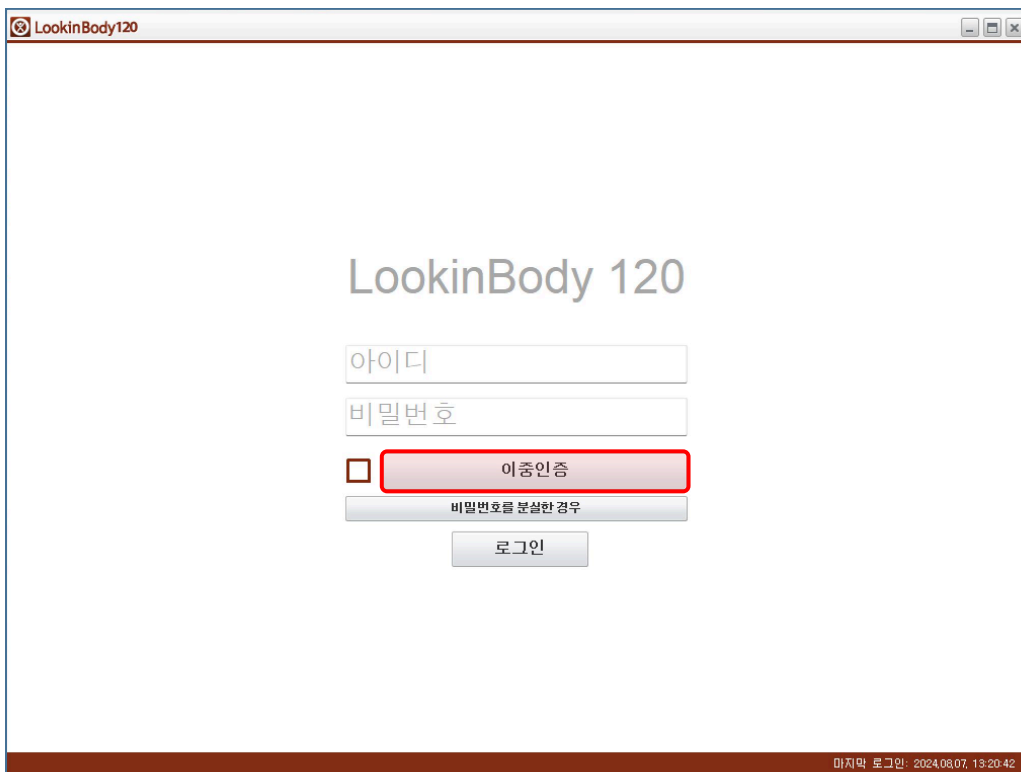
The screenshot shows a web browser window titled "LookinBody120". The main content area displays the "LookinBody 120" logo at the top. Below the logo are two input fields: "아이디" (ID) and "비밀번호" (Password). Underneath these fields is a checkbox labeled "이중인증" (Two-Factor Authentication), which is highlighted with a red rectangular border. Below the checkbox is a button labeled "비밀번호를 분실한 경우" (If you have lost your password). At the bottom of the form is a "로그인" (Login) button. The browser's status bar at the bottom right shows the text "마지막 로그인: 2024.08.07. 13:20:42".

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The screenshot shows the LookinBody120 web application interface. The main content area is divided into three sections: '회원 선택' (Member Selection), '검사 선택' (Test Selection), and '결과 관리' (Result Management). The '회원 선택' section contains a search dropdown menu with the text '성명 또는 회원번호로 검색' (Search by name or member number), which is highlighted with a red box. Below it are input fields for '인바디 검사일로 검색' (Search by InBody test date) and '신규회원등록' (New member registration). The '검사 선택' section has a button for '인바디 검사' (InBody test). The '결과 관리' section has buttons for '결과지인쇄' (Print results), '이메일보내기' (Send email), and '데이터편집' (Edit data). Below these sections is a table titled '회원 목록' (Member list) with columns for '선택' (Select), '성명' (Name), '회원번호' (Member number), '신장 (cm)' (Height), '연령' (Age), '성별' (Gender), '회원정보' (Member info), and '웹스리포트' (Web report). The table is currently empty. To the right of the table is a '사용 안내' (Usage guide) section with the text '회원을 먼저 선택하세요.' (Please select a member first.) and a note that selecting a member allows the use of '검사 선택' and '결과 관리' (Test selection and Result management) menu items. At the bottom of the page, there are links for '인바디와 연결중...' (Connecting to InBody...), '연결 문제 해결' (Troubleshooting connection issues), and '로그아웃' (Logout), along with the version number 'Ver.LB120.5.0.0.1(004)'.

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The screenshot shows the LookinBody120 web application interface. The main content area displays a table of search results for members. The table has columns for selection, name, membership number, height, age, gender, membership info, and health report. One member is selected, and the row is highlighted with a red border.

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Additional interface elements include search filters on the left, a '인바디 검사' button, and a '결과 관리' section on the right. The bottom of the page contains the text '통신실패', '연결 문제 해결', and 'Ver.LB120.5.0.0.1(004)'.

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The screenshot shows the 'Lookin' Body120' software interface. The '회원 선택' (Member Selection) tab is active. Under this tab, there are three search options: '병력으로 검색' (Search by Medical History), '당뇨' (Diabetes), and '인바디 검사일로 검색' (Search by InBody Test Date). The '병력으로 검색' option is highlighted with a red box, and the '검색' (Search) button next to it is also highlighted. Below the search options, there is a table with the following data:

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The screenshot shows the 'LookinBody120' application window. The '회원 선택' (Member Selection) tab is active. In the '그룹으로 검색' (Search by Group) section, a dropdown menu is open, showing '테스트그룹1' selected. Below this, there is a table with one member listed:

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At the bottom of the window, the status bar shows 'InBody270 : 통신성공 (Bluetooth : INBODY270-TEST0513K)' and 'Ver.LB120.3.0.1.3'.

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The screenshot shows the 'LookinBody120' software interface. The main area is titled '회원 선택' (Member Selection). It features search filters for name/member number, in-body test date, and a date range (2017.08.13 - 2017.08.14). A large '인바디 검사' (In-body Test) button is prominent. Below the filters is a table of members. The first row is highlighted in red, indicating a selected member. The table columns are: 선택 (Select), 성명 (Name), 회원번호 (Member No.), 신장 (cm) (Height), 연령 (Age), 성별 (Gender), 회원정보 (Member Info), and 헬스리포트 (Health Report). The selected member has the name '인바디770', member number 'inbody770', height '182.118', age '35', and gender '남자' (Male). The '회원정보' and '인바디' buttons for this member are also highlighted. To the right, there is a '결과 관리' (Result Management) section with buttons for '결과지인쇄' (Print Results), '이메일보내기' (Send Email), and '데이터편집' (Edit Data). A '사용 안내' (Usage Guide) section is also visible, advising users to select members first.

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
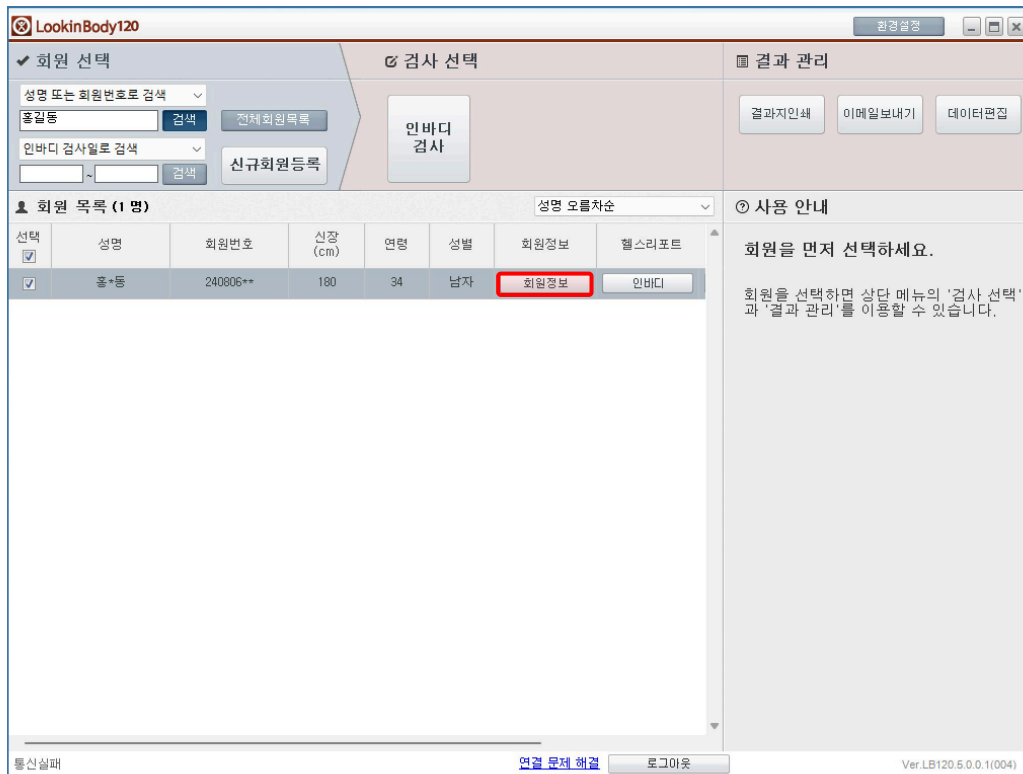
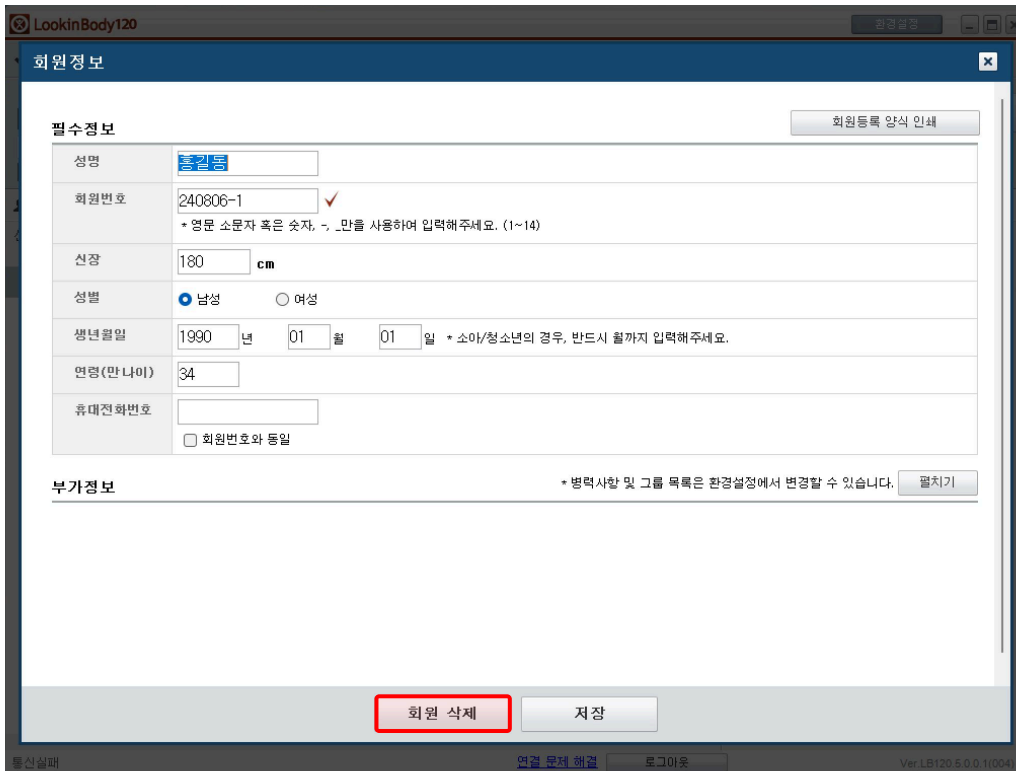
The screenshot shows the '회원정보' (Member Information) form in the LookinBody120 application. The form is divided into '필수정보' (Required Information) and '부가정보' (Additional Information) sections. The '필수정보' section includes fields for name (홍길동), member ID (240806-1), height (180 cm), gender (Male), birth date (1990-01-01), and age (34). The '부가정보' section has a checkbox for '회원번호와 동일' (Same as member ID) and a '펼치기' (Expand) button. At the bottom of the form, there are two buttons: '회원 삭제' (Delete Member) and '저장' (Save). The '회원 삭제' button is highlighted with a red box.

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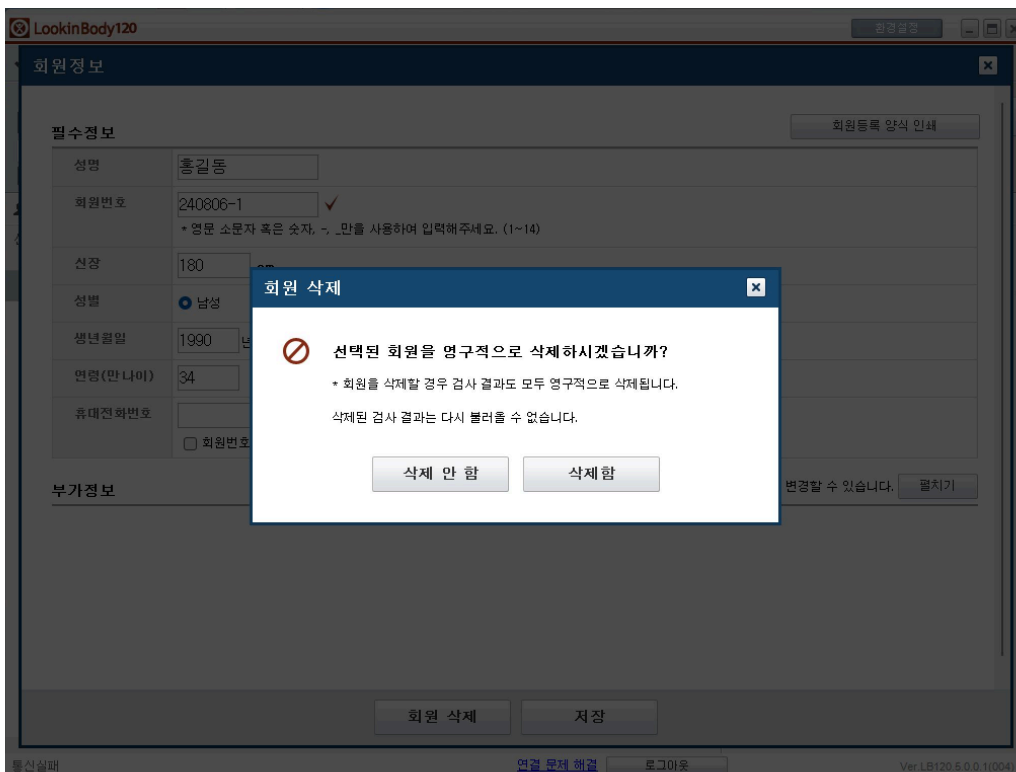The screenshot shows the same '회원정보' form as in the previous image, but with a confirmation dialog box overlaid. The dialog box is titled '회원 삭제' (Delete Member) and contains the following text: '선택된 회원을 영구적으로 삭제하시겠습니까?' (Are you sure you want to permanently delete the selected member?). Below this, it states: '* 회원을 삭제할 경우 검사 결과도 모두 영구적으로 삭제됩니다.' (When deleting a member, all test results will also be permanently deleted.) and '삭제된 검사 결과는 다시 불러올 수 없습니다.' (Deleted test results cannot be reloaded). At the bottom of the dialog box, there are two buttons: '삭제 안 함' (Do not delete) and '삭제 함' (Delete). The '회원 삭제' button in the background form is still visible but dimmed.

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The screenshot displays the LookinBody120 web application interface. The main content area is divided into three sections: '회원 선택' (Member Selection), '검사 선택' (Test Selection), and '결과 관리' (Result Management). The '회원 선택' section includes search filters for name/ID, phone number, and ID card number, along with a search button and a '전체 회원목록' (All Member List) button. The '검사 선택' section features a '인바디 검사' (InBody Test) button. The '결과 관리' section contains buttons for '결과지 인쇄' (Print Results), '이메일보내기' (Send Email), and '데이터편집' (Data Edit), with the latter highlighted by a red box. Below these sections is a table titled '회원 목록 (1명)' (Member List (1 member)) with columns for selection, name, ID number, height, age, gender, ID card number, and InBody test status. A single member is listed with the name '홍*동', ID '240806**', height '180', age '34', and gender '남자'. The '선택' column has a checked checkbox, and the '인바디' column has a button labeled '회원정보' (Member Info). To the right of the table is a '사용 안내' (Usage Guide) section with the text: '회원을 먼저 선택하세요. 회원을 선택하면 상단 메뉴의 '검사 선택' 과 '결과 관리'를 이용할 수 있습니다.' (Please select a member first. After selecting a member, you can use the 'Test Selection' and 'Result Management' menus in the top menu.) The bottom of the interface shows a status bar with '통신실패' (Communication failed), '연결 문제 해결' (Resolve connection problem), '로그아웃' (Logout), and the version 'Ver.LB120.5.0.0.1(004)'.

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The screenshot shows the LookinBody120 software interface. The window title is 'LookinBody120'. The interface is divided into several sections:

- 회원 선택 (Member Selection):** Includes search fields for '성명 또는 회원번호로 검색' (Search by name or member number) and '인바디 검사일로 검색' (Search by device check date), along with buttons for '검색' (Search), '전체회원목록' (All member list), and '신규회원등록' (New member registration).
- 검사 선택 (Check Selection):** Contains two buttons: '인바디 검사' (InBody Check) and '혈압 검사' (Blood Pressure Check), with the latter highlighted by a red box.
- 결과 관리 (Result Management):** Includes buttons for '결과지인쇄' (Print result), '이메일보내기' (Send email), and '데이터편집' (Edit data).
- 회원 목록 (1명) (Member List (1 member)):** A table with columns for '선택' (Select), '성명' (Name), '회원번호' (Member number), '신장 (cm)' (Height), '연령' (Age), '성별' (Gender), '회원정보' (Member info), and '헬스리포트' (Health report). One member is listed: '홍길동' (Hong Gil-dong) with member number 'hongm', height '190.5', age '35', and gender '남자' (Male).
- 사용 안내 (Usage Guide):** Contains the text: '회원을 먼저 선택하세요.' (Please select a member first.) and '회원을 선택하면 상단 메뉴의 '검사 선택' 과 '결과 관리'를 이용할 수 있습니다.' (After selecting a member, you can use 'Check Selection' and 'Result Management' in the top menu.)

At the bottom left, it says 'InBody270 : 통신성공 (Bluetooth : INBODY270-TEST0513K; 혈압계)'. At the bottom right, it says 'Ver.LB120.3.0.1.3'.

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
The screenshot shows the LookinBody120 software interface. The main window is titled 'LookinBody120' and contains several sections:

- 회원 선택 (Member Selection):** Includes search fields for name/member ID and InBody ID, and buttons for '검색' (Search), '전체회원목록' (All member list), and '신규회원등록' (New member registration).
- 검사 선택 (Test Selection):** Features two buttons: '인바디 검사' (InBody test) and '혈압 검사' (Blood pressure test), with the latter highlighted in red.
- 결과 관리 (Result Management):** Contains buttons for '결과지인쇄' (Print result), '이메일보내기' (Send email), and '데이터편집' (Edit data).
- 선택 회원 목록 (1명) (Selected member list (1 person)):** A table with columns for selection, name, member ID, height, age, gender, member info, and health report. One member, '홍길동' (Hong Gil-dong), is listed with ID 'hongm', height '190.5', age '35', and gender '남자' (Male).
- 사용 안내 (User Guide):** A text box stating: '혈압계에서 [시작] 버튼을 누르면 혈압 검사가 시작됩니다.' (When you press the [Start] button on the blood pressure monitor, the blood pressure test begins.)

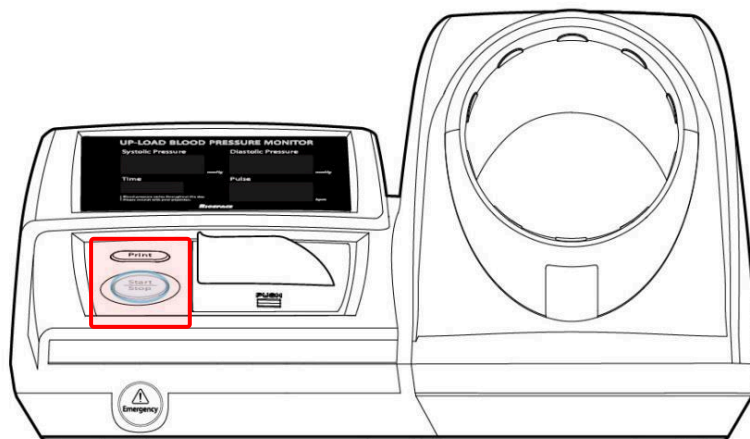At the bottom of the interface, there is a footer with the text: 'InBody270 : 통신성공 (Bluetooth : INBODY270-TEST0513K, 혈압계) Ver.LB120.3.0.1.3'.

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The screenshot shows the LookinBody120 web application interface. It is divided into three main sections: '회원 선택' (Member Selection), '검사 선택' (Test Selection), and '결과 관리' (Result Management). The '회원 선택' section includes search filters for name or ID, and a table of members. The '검사 선택' section has a large '인바디 검사' (InBody Test) button. The '결과 관리' section contains buttons for '결과지 인쇄', '이메일보내기', and '데이터편집'. Below the member table, there is a '사용 안내' (Usage Guide) section with instructions on how to use the '인바디' button.

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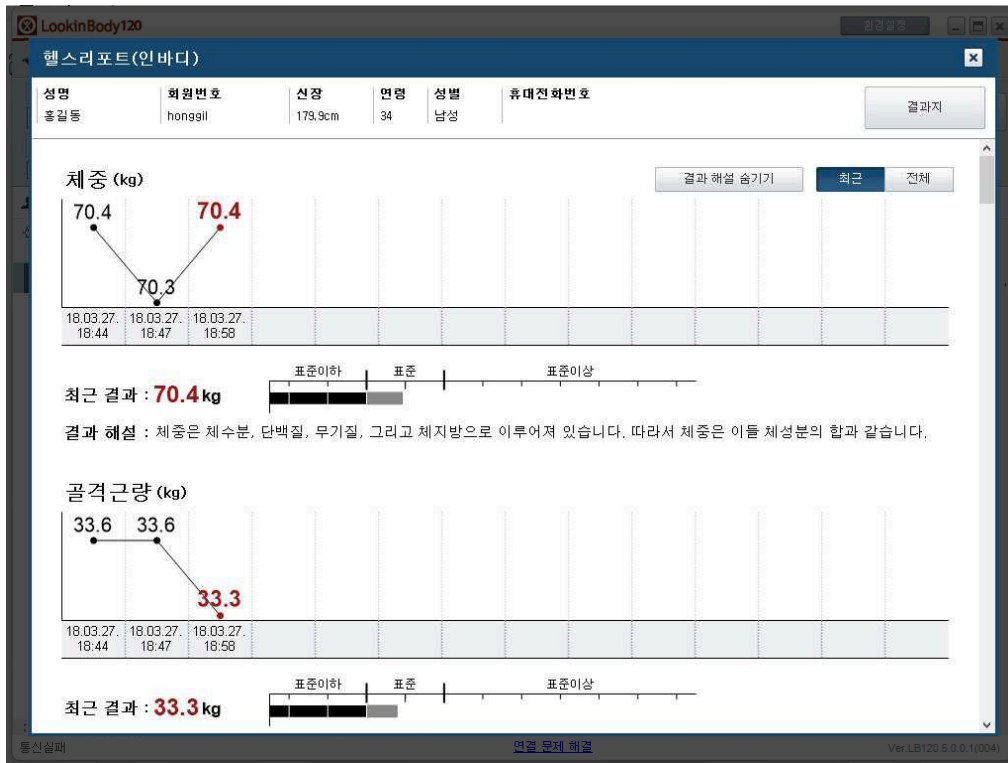
The screenshot shows the '인바디결과지' (InBody Results) page in the LookInBody120 software. At the top, there is a header with the user's name '홍길동', ID 'honggil', height '179.9cm', age '34', and gender '남성'. Below this, there are two line graphs: '체중 (kg)' (Weight) and '골격근량 (kg)' (Skeletal Muscle Mass). The weight graph shows values of 69.0, 70.4, 70.3, and 70.4 kg over time. The skeletal muscle mass graph shows values of 37.5, 33.6, 33.6, and 33.3 kg. A sidebar on the right contains buttons for '결과지', '인바디결과지' (highlighted in red), '체수분결과지', and '신체변화결과지'. Below the graphs, there are '최근 결과' (Recent Results) sections for weight (70.4 kg) and skeletal muscle mass (33.3 kg), each with a progress bar indicating the user's status relative to standard ranges (표준이하, 표준, 표준이상).

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The screenshot shows a comparison of two InBody results side-by-side. Both results are for the same user (ID: honggil, height: 179.9cm, age: 34, gender: 남성) at the same time (2018.03.27, 18:44:49 and 18:47:16). The left result shows a weight of 70.4 kg and skeletal muscle mass of 33.6 kg. The right result shows a weight of 70.3 kg and skeletal muscle mass of 33.6 kg. Both results also show body fat mass of 10.4 kg. Below the main data, there are detailed analysis sections: '체성분분석' (Body Composition Analysis), '골격근·지방분석' (Muscle-Fat Analysis), and '비만분석' (Obesity Analysis). Each section includes a progress bar and numerical values for various metrics like BMI, body mass index, and percent body fat.

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The screenshot shows the 'LookinBody120' web application interface. It is divided into several sections:

- 회원 선택 (Member Selection):** Includes search fields for name/member number, a search button, and a '진체회원목록' (Current Member List) button. There is also a '신규회원등록' (New Member Registration) button.
- 검사 선택 (Test Selection):** Features a large '인바디 검사' (InBody Test) button.
- 결과 관리 (Result Management):** Contains buttons for '결과지 인쇄' (Print Results), '이메일보내기' (Send Email), and '데이터편집' (Edit Data).
- 회원 목록 (1명) (Member List (1 member)):** A table with columns for selection, name, member number, member info, and health reports. The '혈압' (Blood Pressure) button is highlighted with a red box.

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At the bottom of the page, there is a footer with '통신실패' (Communication failed), a link to '연결 문제 해결' (Resolve connection issues), and the version number 'Ver.LB120.5.0.0.1(004)'.

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The screenshot shows the LookinBody120 web application interface. It is divided into several sections:

- 회원 선택 (Member Selection):** Includes search filters for name/member number, gender, and age, along with search buttons and a 'New Member Registration' button.
- 검사 선택 (Test Selection):** Features a large '인바디 검사' (InBody Test) button.
- 결과 관리 (Result Management):** Contains buttons for '결과지 인쇄' (Print Results), '이메일보내기' (Email), and '데이터편집' (Data Edit).
- 회원 목록 (1명) (Member List):** A table with columns for selection, name, member number, member info, and health report. The '혈당' (Blood Sugar) button in the health report column is highlighted with a red box.

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At the bottom of the page, there is a footer with '통신실패' (Communication failed), a link to '연결 문제 해결' (Resolve connection problem), and the version number 'Ver.LB120.5.0.0.1(004)'.

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The screenshot shows the LookinBody120 web application interface. The top navigation bar includes '회원 선택' (Member Selection), '검사 선택' (Test Selection), and '결과 관리' (Result Management). The '결과 관리' section contains three buttons: '결과지인쇄' (Print Results), '이메일보내기' (Send Email), and '데이터편집' (Data Edit), which is highlighted with a red box. Below the navigation bar, there is a table titled '회원 목록 (1 명)' (Member List (1 member)) with columns for '선택' (Select), '성명' (Name), '회원번호' (Member No.), '신장 (cm)' (Height), '연령' (Age), '성별' (Gender), '회원정보' (Member Info), and '헬스리포트' (Health Report). The table contains one row for a member named '홍*동' with member number '240806**', height '180', age '34', and gender '남자' (Male). The '회원정보' and '인바디' buttons are visible next to the member's name. The right sidebar contains a '사용 안내' (Usage Guide) section with the text: '회원을 먼저 선택하세요. 회원을 선택하면 상단 메뉴의 '검사 선택' 과 '결과 관리'를 이용할 수 있습니다.'

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The screenshot shows a dialog box titled "11. GDPR 옵션" with a close button in the top right corner. It contains several sections with checkboxes and buttons:

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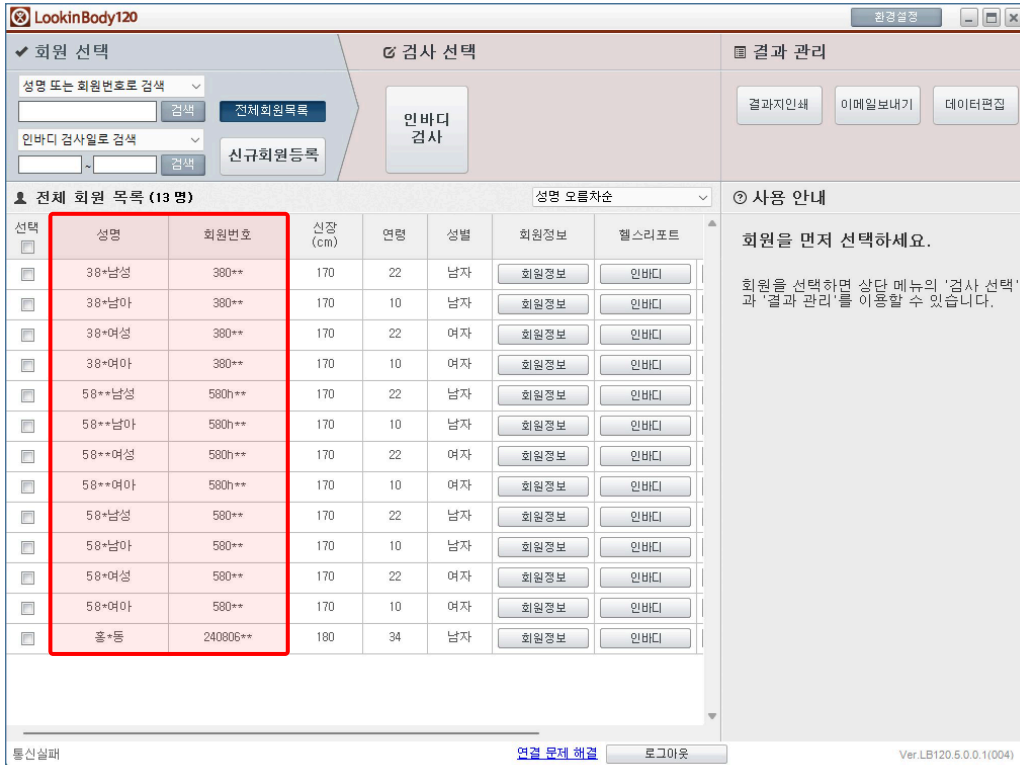
At the bottom center, there is a "저장" (Save) button, which is highlighted with a red box.

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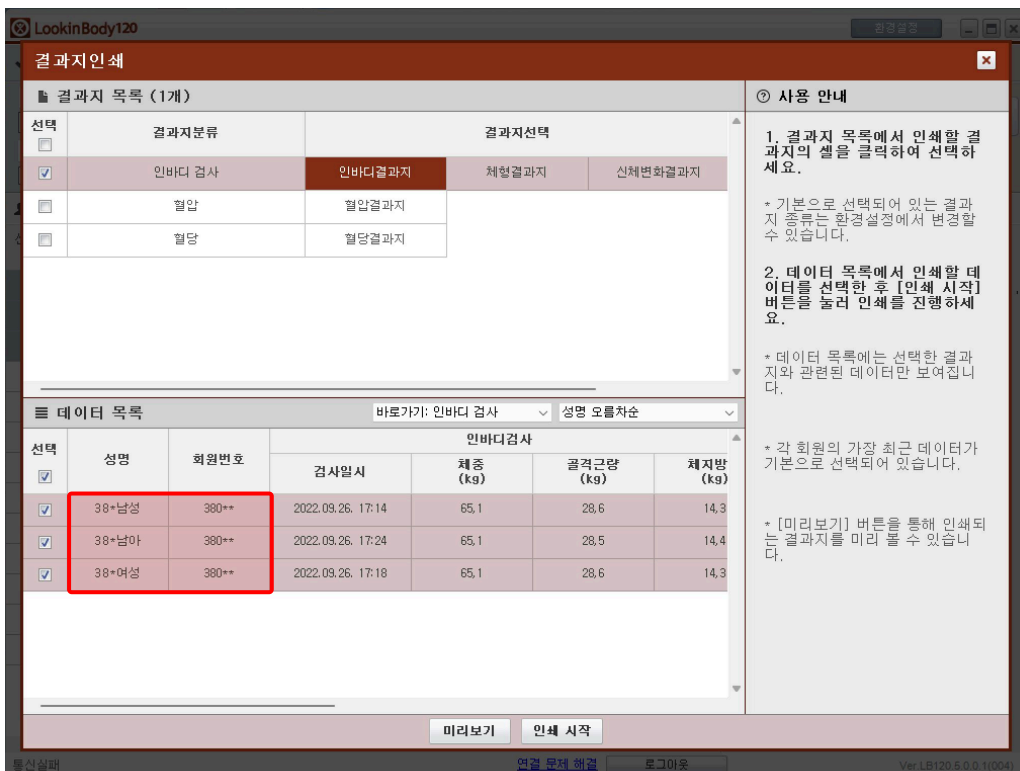
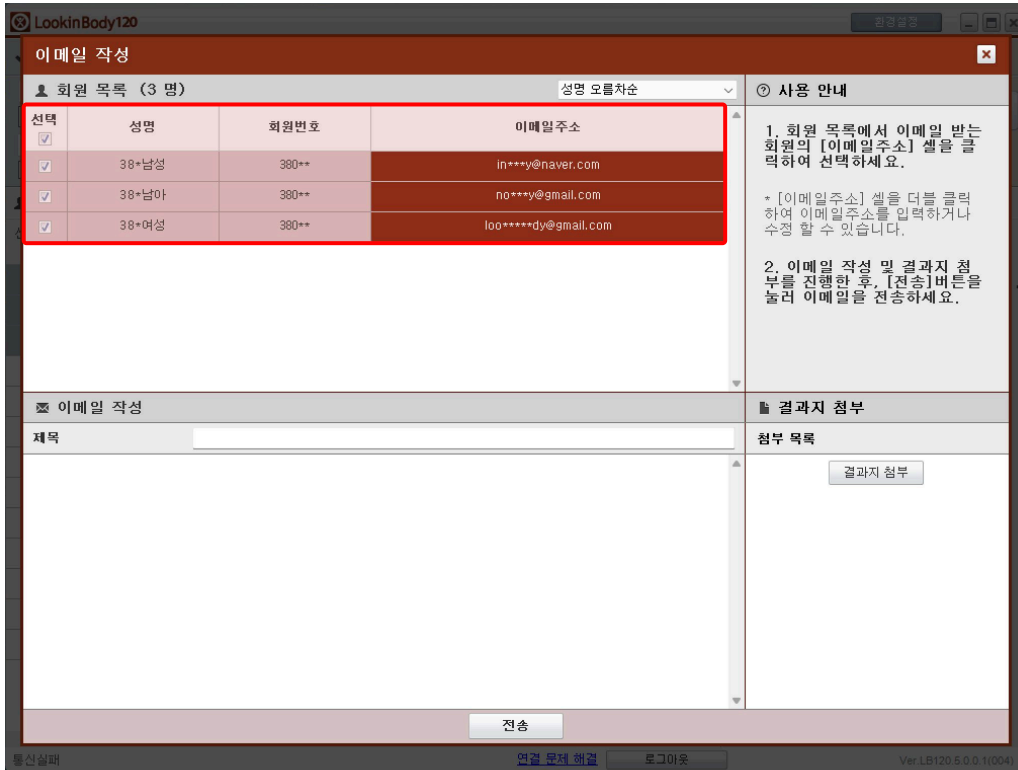
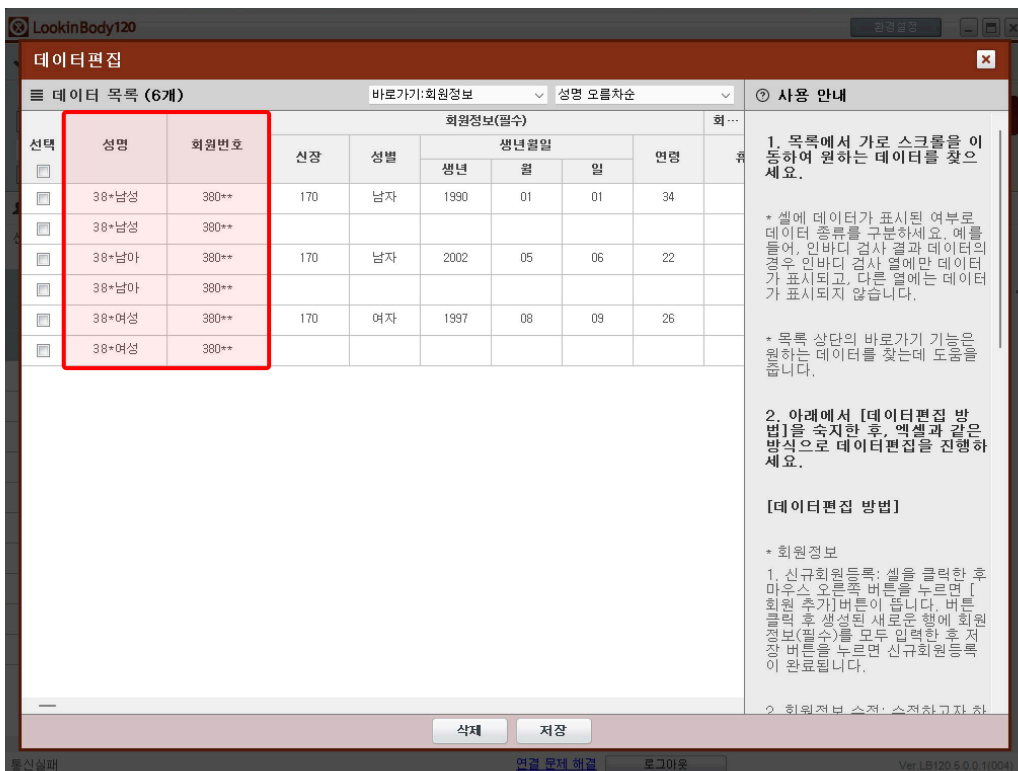
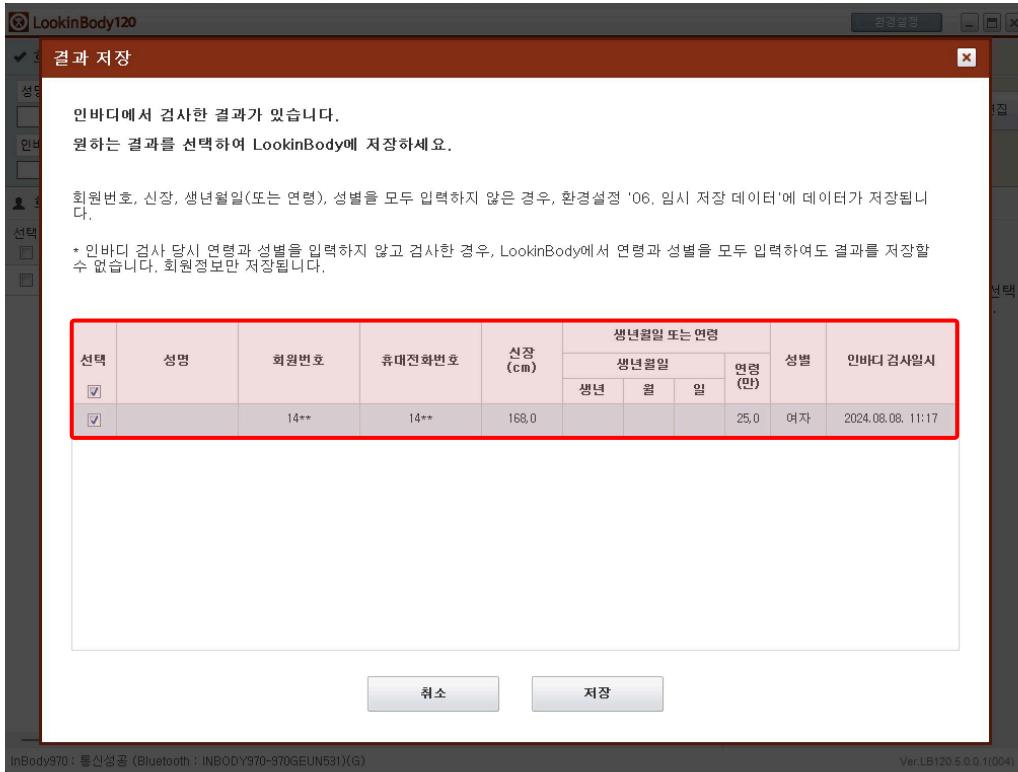
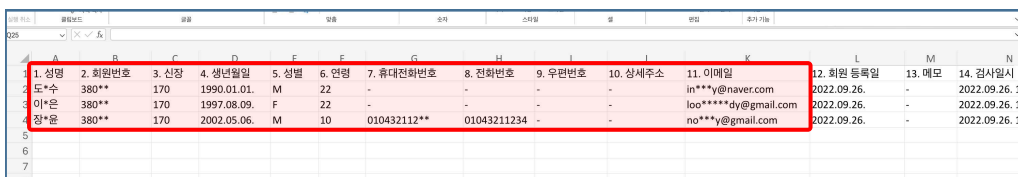
The screenshot shows the 'LookinBody120' web application interface. The main area is divided into three sections: '회원 선택' (Member Selection), '검사 선택' (Test Selection), and '결과 관리' (Result Management). The '회원 선택' section includes search filters for name/member number and member ID, along with buttons for '전체회원목록' (All Member List) and '신규회원등록' (New Member Registration). The '검사 선택' section features a large '인바디 검사' (InBody Test) button. The '결과 관리' section has buttons for '결과지인쇄' (Print Results), '이메일보내기' (Send Email), and '데이터관급' (Data Management). Below these sections is a table titled '전체 회원 목록 (13 명)' (All Member List (13 Members)) with columns for selection, name, member number, member info, and health reports. The '헬스리포트' (Health Report) column contains buttons for '인바디' (InBody), '혈압' (Blood Pressure), and '혈당' (Blood Sugar). The first row of the table has these three buttons highlighted with red boxes. To the right of the table is a '사용 안내' (Usage Guide) section with the text '회원을 먼저 선택하세요.' (Please select a member first.) and a note about using the '검사 선택' (Test Selection) and '결과 관리' (Result Management) options after selecting a member. The footer of the interface includes '인바디와 연결중...' (Connecting to InBody...), a link to '연결 문제 해결' (Troubleshooting Connection Issues), and the version number 'Ver.LB120.5.0.0.1(004)'.

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The image shows a dialog box with a dark red title bar containing the text "03. 인바디 결과지 연동" and a close button. The main content area is white and contains the following elements:

- The text "인바디 결과지 전송" is positioned at the top left.
- The text "데이터 연동 방식" is positioned below it.
- A dropdown menu is open, showing three options: "사용안함", "사용안함", and "FTP (파일 전송 프로토콜)". The second "사용안함" option is currently selected and highlighted in blue.
- A "저장" (Save) button is located at the bottom center of the dialog box.

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©2025 InBody Co., Ltd. All rights reserved.