# **BSM270/BSM270B**

## **Language Contents**

English	3
한국어	35

# **BSM270/BSM270B**

## **User's Manual**

English

## InBody User's Manual for Measurement Guide and Setup

Thank you for purchasing the InBody.  
This user's manual describes all the features of the InBody.  
Please read before use and keep it in a safe place.  
By following the manual instructions,  
you will be able to use the InBody more safely and effectively.

### Please note the important information below before reading this manual.

#### **Warning**

Failure to comply with safety warnings and regulations can cause serious injury or death.

#### **Caution**

Failure to comply with safety cautions and regulations can cause injury or property damage.

#### **Note**

Referring to notes can help improve equipment use.

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## Representative & Sponsor Information

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#### InBody Oceania [AUSTRALIA]

Main office: Level 8, 1 York Street, SYDNEY, NSW 2000, Australia  
Showroom: U2/82-86 Minnie Street, Southport, Queensland  
TEL: +61-7-5681-1900  
Website: au.inbody.com Email: oceania@inbody.com

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#### InBody India [INDIA]

57/57 A, 1st Floor, Raj Industrial Complex, Military Road, Marol,  
Andheri (East), Mumbai- 400059, Maharashtra, India  
TEL: +91-22-6223-1911  
Website: inbody.in E-mail: india@inbody.com

## Copyright

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This manual may be printed incorrectly and subject to change without notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual. Visit our website (inbody.com) to view and download additional information about the InBody.

InBody Co., Ltd. reserves the right to modify the appearance, specifications, etc. of this product to improve its quality, without prior notice.

©2024 InBody Co., Ltd. All rights reserved.

# BSM270/BSM270B

## User's Manual Table of Contents

### I. BSM270/BSM270B Installation

A. Product Components	6
B. Operating environment	7
C. Installation Instructions	7
D. Setup	13
E. Precautions for Maintenance	16

### II. Measurement

A. Preparatory Steps	17
B. Measurement instructions	17
C. Measuring posture	20

### III. Transportation and Storage

A. Repacking Instructions	21
B. Transportation and Storage Environment	24

### IV. FAQ

### V. Others

A. Safety Information	27
B. Exterior and Functions	30
C. Indication symbols	32
D. Product specifications	33

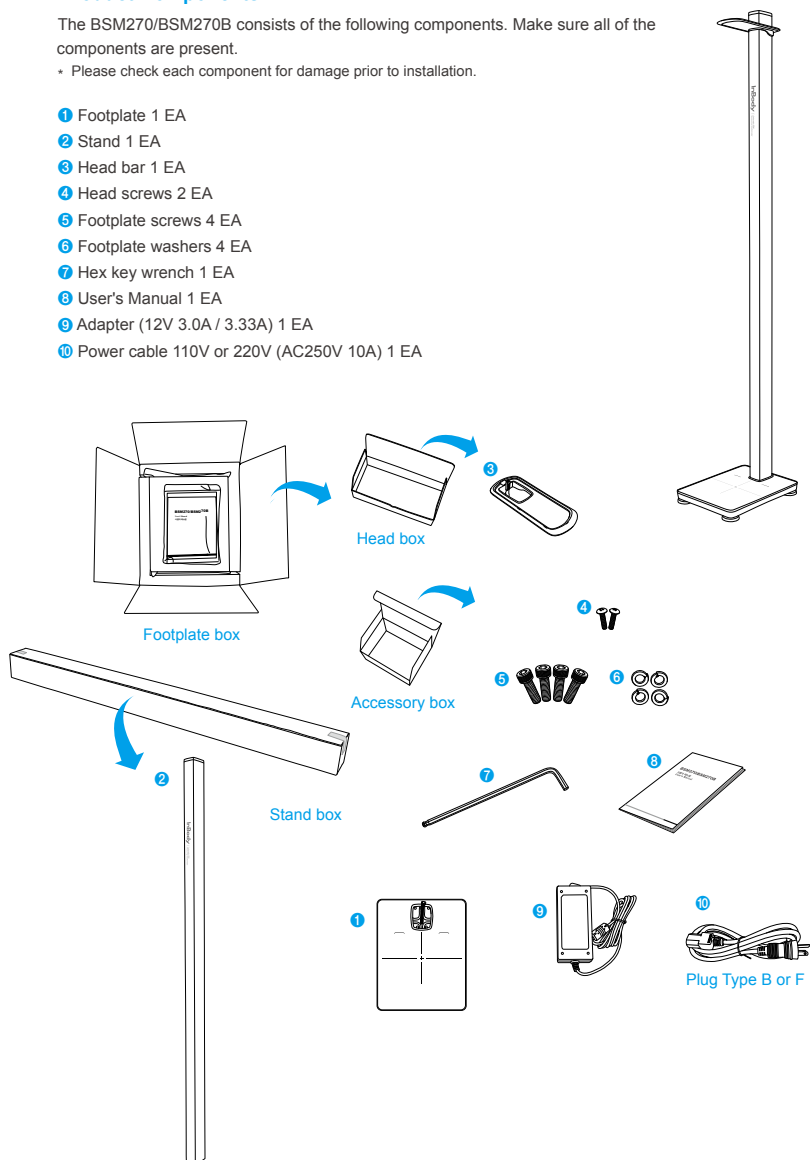
# I. BSM270/BSM270B Installation

## A. Product Components

The BSM270/BSM270B consists of the following components. Make sure all of the components are present.

• Please check each component for damage prior to installation.

- ❶ Footplate 1 EA
- ❷ Stand 1 EA
- ❸ Head bar 1 EA
- ❹ Head screws 2 EA
- ❺ Footplate screws 4 EA
- ❻ Footplate washers 4 EA
- ❼ Hex key wrench 1 EA
- ❽ User's Manual 1 EA
- ❾ Adapter (12V 3.0A / 3.33A) 1 EA
- ❿ Power cable 110V or 220V (AC250V 10A) 1 EA

## B. Operating environment

Please make sure the environment is appropriate for the BSM270/BSM270B installation. This equipment is designed for indoor use. If installing the equipment outdoors, the following requirements must be fulfilled.

Temperature range	10 ~ 40°C (50 ~ 104°F)
Relative humidity	30 ~ 75% RH
Atmospheric pressure range	70 ~ 106kPa

\* Two or more people are required when installing the product.

### WARNING

Before installing the BSM270/BSM270B unit, make sure that the ceiling height is 7.2ft or higher. The height of the equipment is 7.1ft. Please secure a minimum of 1.2in space between the ceiling and the equipment.

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## C. Installation Instructions

1. After unpacking the footplate box, remove the packing material and take out the footplate, head bar, and the other components.

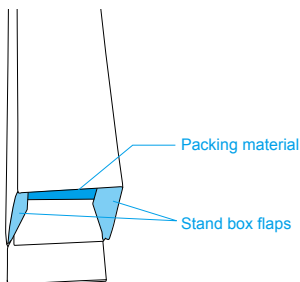
### CAUTION

- Keep the packing material to repack the equipment in the future. Dispose of trash according to local laws and regulations.
- Install the BSM270/BSM270B on a flat, vibration-free surface. If the equipment is installed on a surface that is not flat, it may topple during testing or the results may be inaccurate.
- Do not wash the BSM270/BSM270B unit with detergent. If the detergent flows inside BSM270/BSM270B unit, it can cause corrosion and lead to malfunction. Clean the exterior of the equipment gently with a lint free cloth.

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
2. After unpacking the stand box, remove the packing material and take out the stand.
3. Remove the packing materials from both ends and put them back into the stand box.
4. As shown in the figure below, fold the box flaps inside leaving one flat.

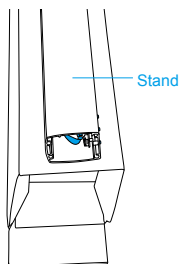5. Place the stand on the stand box so that the assembled screw head is facing right.


#### **CAUTION**

Make sure the cable is properly connected.

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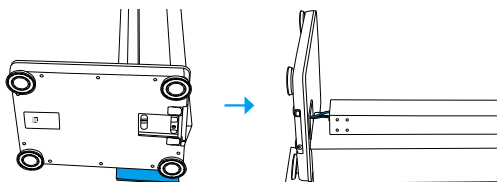
6. Place the footplate on the flat stand box flap and connect the footplate cable to the stand.

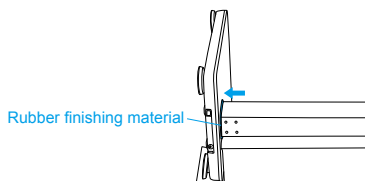7. Hold the footplate securely and insert the stand into the groove of the footplate.


**CAUTION**

Please be careful when inserting the stand as the measured values may be inaccurate if the rubber finishing material is inserted along with the stand.

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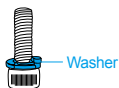
8. Insert the footplate washer into the footplate screw.

9. Insert the four screws including the washers into the four screw grooves on the back of the footplate.

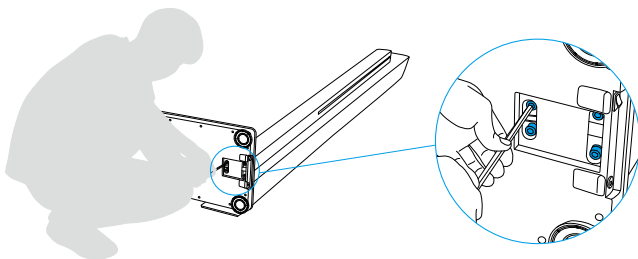
10. Screw clockwise to tighten with the hex key wrench.

11. Once the stand is firmly assembled on the footplate, pull it up to the upright position.

12. Tilt the assembled equipment sideways to align the groove of the head bar with the stand.

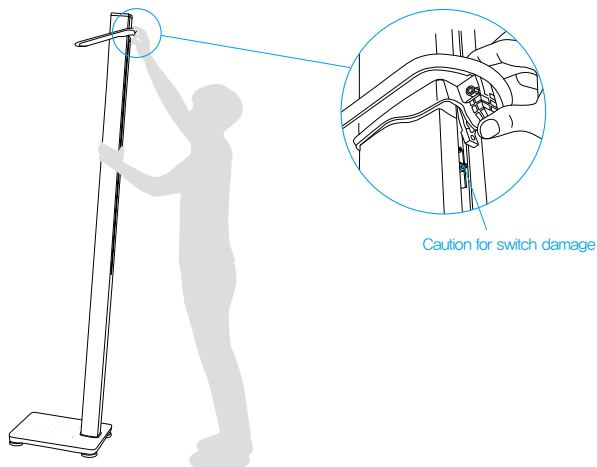Then tilt the head bar as shown in the image below, and insert it to the location of the screw groove without damaging the switch.


### **CAUTION**

- Inserting the head bar without tilting it may damage the equipment and cause malfunction. Make sure to tilt the head bar.
- Do not lower the drive component with force. This may cause malfunction.

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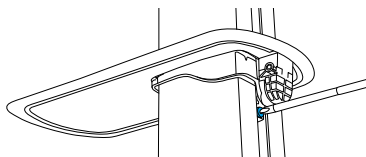13. Align the grooves in the stand screws with the grooves in the head bar screws, and then insert the head screws. Use the Phillips screwdriver to rotate the head screws clockwise to secure it tightly.


### **CAUTION**

Avoid using excessive force to rotate the screws, which may damage the screws.

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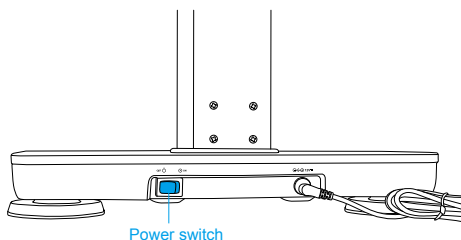14. Connect the adapter and the power cable, and plug into the input for power source in the back of the footplate.

### WARNING

- Do not install the equipment where the power cable is difficult to disconnect.
- To avoid electric shock, do not plug or disconnect the power cable with wet hands.
- Always use an outlet designed for the correct power supply (AC100-240 V).  
An incorrect power supply may cause fire or equipment malfunction.
- If using a multiple tab, ensure it has a sufficient power capacity or use an extension cable.
- Do not disassemble the equipment arbitrarily. This may cause electric shock or injury, product malfunction, and/or inaccurate results. Unauthorized disassembly will void the manufacturer's warranty.
- Do not let the BSM270/BSM270B touch other electronic devices when turned on. This may cause electric shock.

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### CAUTION

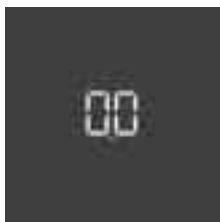
- External interface inputs are located at the rear bottom of the equipment, which may allow liquids or foreign objects to flow inside the equipment. Please note that this may cause critical damage to the electronic components of the equipment.
- Make sure to use the power adapter provided by InBody.
- When connecting the adapter cable to the BSM270/BSM270B unit, insert the adapter cable completely into the power input terminal.
- Please note that the weight may vary when measuring at an altitude of 2,000m or higher.
- Do not support yourself on the equipment when stepping up or down from the footplate.
- Please be careful not to trip or get your foot caught in the footplate.

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
15. Turn on the BSM270/BSM270B by referring to the image in step 14.

16. The head will operate once and the screen on footplate will change as below image, as a standby mode.

<Without bluetooth function>

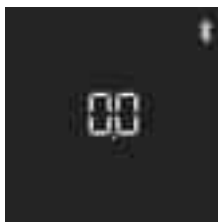
<With bluetooth function>

## D. Setup
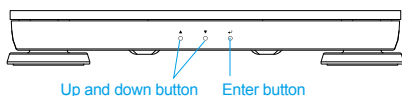
### 1. How to set up

In setup, you can set the measurement mode, offset, unit, head bar position, display orientation, automatic height measurement, display time, weight measurement time, and equipment level check.

- 1) Press the Enter button on the base for more than three seconds, and the setting screen appears as shown in the upper right.
- 2) Use the up and down buttons (▲, ▼) to move to the desired setting item.
- 3) Press the Enter button to select the item.
- 4) To change additional setup items, press up and down buttons (▲, ▼) to move to the other setting items. Once the changes are made, press the Enter button for more than 3 seconds to save the changes and go back to standby mode.

### 2. Setup Menu

#### 1) **mod** (Mode)

You can select Mode depending on the purpose of use:

- ① **W.H** : Select when measuring both weight and height.
- ② **W** : Select when measuring weight only.
- ③ **H** : Select when measuring height only.

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#### 2) **OFFS** (Offset)

You can adjust the offset values for weight and height. Extra weight or height due to clothing or accessories can be adjusted when offset values are set.

- ① **W** : Adjust the weight offset value. (Adjustment range: +5.0 to -5.0 kg, fixed unit: 0.1 kg / Adjustment range: +10.0 lb to -10.0 lb, adjustment unit: 0.2 lb)
- ② **H** : Adjust the height offset value. (Adjustment range: +50.0 to -50.0 cm, adjustment unit: 0.1 cm)

\*Height offset function is available only in centimeters.

3) **Unit** (Unit)

Select the measurement unit. You can choose between kg/cm and lb/in.

4) **Head Position**

Set the head bar position. You may choose between **Bar** and **HF**.

When you change the settings, - - - will appear and return to **Head** after completed.

1 **Bar** : Select when measuring heights at 1m60cm (5'3") or higher.

2 **HF** : Select when measuring heights at 1m60cm (5'3") or lower.

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5) **Display** (Display)

Set the display orientation.

1 **Bar** : Select when the examinee is facing towards the display.

2 **Bar** : Select when the operator is facing towards the display.

3 **Bar** : Select when both the examinee and the operator are facing towards the display together.

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6) **Head Mode**

Set the automatic measurement mode.

1 **Bar** : Select to measure height automatically after the weight measurement.

2 **Bar** : Select to measure height by pressing the Enter button after the weight measurement.

7) **Display Sec** (Display Sec)

Set screen timeout duration.

(Adjustment range: 5-60 seconds)

8) **Speed** (Speed)

Set the stabilization time for weight measurement.

1 **Bar** : Select to measure weight when there is no movement.

2 **Bar** : Select to measure weight after 2 seconds even if there is a movement.

9)  (Level Test)

You can check the equipment level by the location of the small square. Level the equipment by rotating the leveling screws under the footplate.

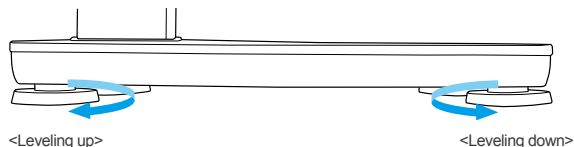
 **NOTE**

If the location of the small square is not centered, rotate the leveling screws on the opposite side of the square to level.

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 **CAUTION**

Avoid injuring your hands when rotating the leveling screws under the footplate.

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## E. Precautions for maintenance

### CAUTION

- Do not disassemble the equipment arbitrarily. This may cause electric shock or product malfunction. There is no parts in the equipment for the users to operate. InBody Co., Ltd. is not responsible for any damage caused by this action.
- For long-term storage, unplug the unit from the outlet.
- Do not move the device while the power is on.
- Do not let foreign objects like dust and liquid enter the device, as they can cause damage to electrical components.
- Do not impact the head bar. The measurement may not be completed.
- Clean the exterior of the equipment gently with a lint-free cloth once a week.
- Dispose of trash according to local laws and regulations.
- Do not pull down the head bar with force. It may cause product malfunction.

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### A. Preparatory steps

#### CAUTION

- Avoid injuring your hand from getting stuck between the base and the stand.
- Examinees with physical disability or children can measure with the help of the operator.
- Level the equipment by screwing the leveling screws if the equipment shakes.

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### B. Measurement instructions

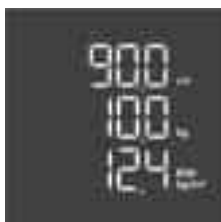
To change the measurement mode, please refer to 1)  6) 

1. Measure height and weight simultaneously, and check BMI.


When the examinee stands on the footplate, weight will be displayed on the screen. Once the weight value is stable, measurement will be completed with a beeping sound. Then, the height is measured.

If it is set to '' (automatic measurement) in the Setup, height will be measured automatically.


If it is set to '' (manual measurement), in the Setup, height will be measured after pressing the 'Enter' button.

<Without bluetooth function>
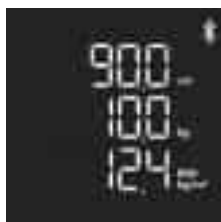
<With bluetooth function>

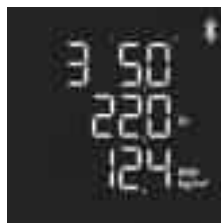
## 2. Measure weight only



When the examinee stands on the footplate, weight will be displayed on the screen. Once the weight value is stable, measurement will be completed with a beeping sound.
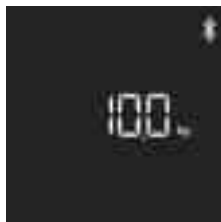
<Without bluetooth function>





<With bluetooth function>
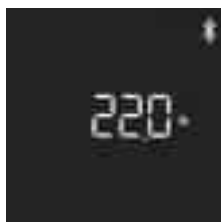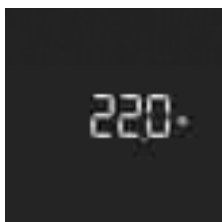
### 3. Measure height only

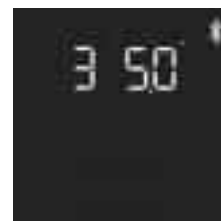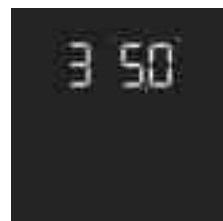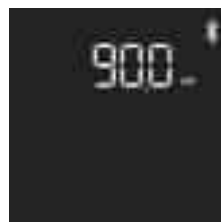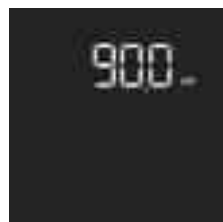Height measurement can be done in two modes: 'Auto' (automatic measurement) and 'MANU' (manual measurement). If the measurement mode 'MANU' in Setup is set to 'Auto' (automatic measurement), height measurement will begin automatically. If it is set to 'MANU' (manual measurement), height measurement will begin after pressing the 'Enter' button.


<Without bluetooth function>

<With bluetooth function>

\* The BSM270B can be connected to InBody device via bluetooth communication.  
Connecting instructions can be found in InBody user's manual.

### C. Measuring posture

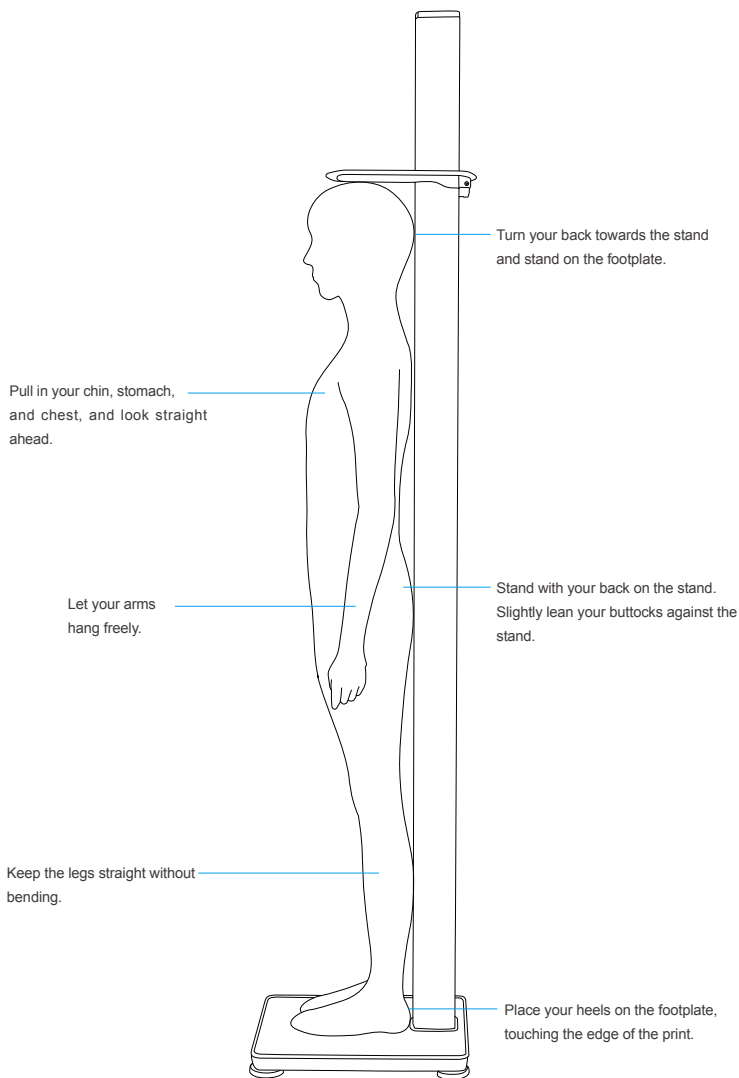The examinee must maintain proper posture to have accurate measurement results.


### III. Transportation and Storage

#### A. Repackaging instructions


Before repackaging the equipment, be sure to turn off the power and remove the adapter.

Avoid damaging the head bar and cables during the repackaging process.

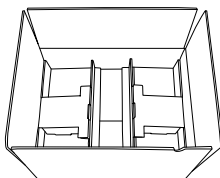1. Prepare the stored BSM270/BSM270B footplate box.
2. Place two paper boxes on both ends of the packing box as shown below.

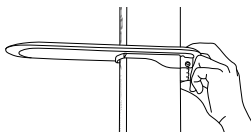3. Turn off the equipment.
4. Hold the back of the head bar and lower it slowly with your hands.

#### WARNING

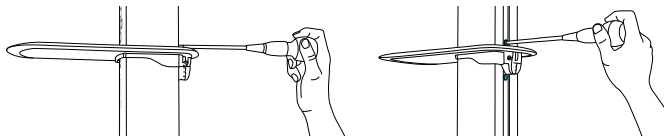
- Lowering the head bar by holding the front of the head bar may damage the equipment. Therefore, please lower it by holding the back of the head bar.
- Lowering the head bar too quickly may damage the equipment. Therefore, please lower it slowly and gently.
- Do not apply force to the head bar unless it is a special case, such as repackaging, as there is a risk of damage.

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
5. Remove the head bar by loosening the head screw with a screwdriver.


6. Insert the head bar into the head box.

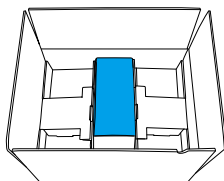7. Put the head box in the footplate packing box.

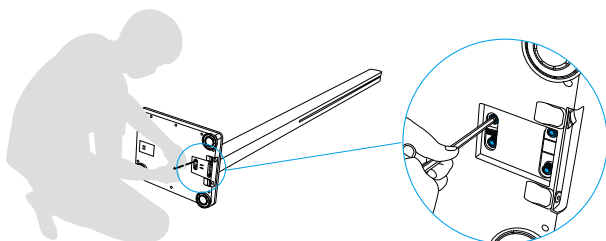
8. Lay the equipment down on the side to disassemble the stand and the footplate.


9. Put the adapter, cable, and screws in the accessory box.

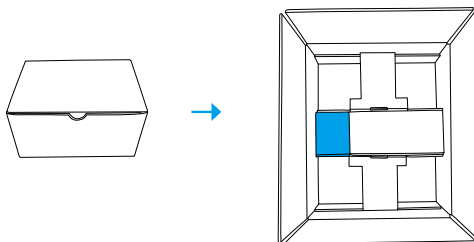10. Put the accessory box in the footplate box.

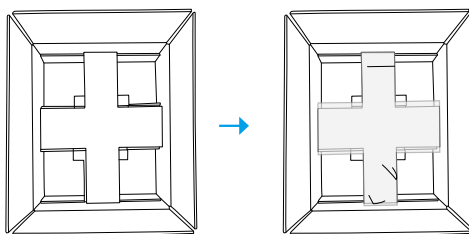
11. Lift the cross-shaped pad and put it in the foam bag.


EN

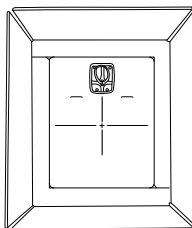
12. Place the polyethylene foam pad and put the disassembled footplate on top as shown below.

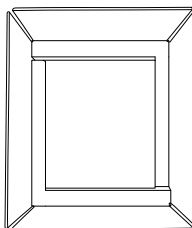13. Place the foam bag on top, and put in the box pad.

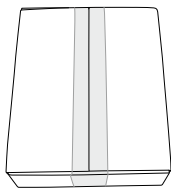
14. Close the base box and seal it with tape.


15. Put the pads in the middle and both ends of the stand.

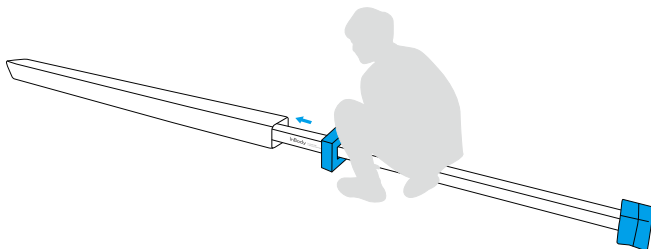16. Insert the stand into the stored stand box.

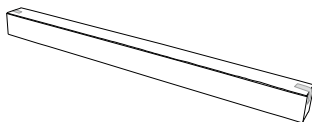17. Close the stand box and seal it with tape.







## B. Transportation and Storage Environment

The BSM270/BSM270B should be transported or stored under the following conditions.

Temperature range	-10 ~ 70°C (14 ~ 158°F)
Relative humidity	10 ~ 80% RH (No Condensation)
Atmospheric pressure range	50 ~ 106kPa
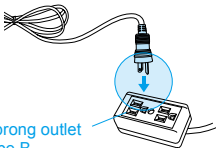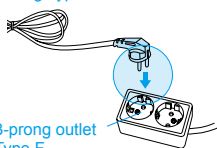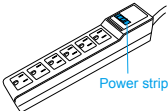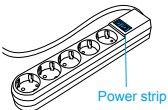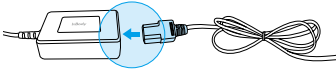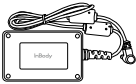
## IV. FAQ

Several commonly asked questions and answers are listed below. If a problem persists, please contact Product Support.

Issue	Potential Solution
• The device is not turning on.	<ul style="list-style-type: none"><li>• Plug the power cable into a properly grounded (3-prong) outlet.</li></ul> <div><div><p>&lt;Plug Type B&gt;</p><p>3-prong outlet Type B</p></div><div><p>&lt;Plug Type F&gt;</p><p>3-prong outlet Type F</p></div></div>
	<ul style="list-style-type: none"><li>• When using a power strip, make sure the switch is turned on.</li></ul> <div><div><p>&lt;Plug Type B&gt;</p><p>Power strip</p></div><div><p>&lt;Plug Type F&gt;</p><p>Power strip</p></div></div>
	<ul style="list-style-type: none"><li>• Plug the power cable completely into the adapter terminal.</li></ul> 
	<ul style="list-style-type: none"><li>• Make sure you are using the power adapter (DC 12V 3.0A/ 3.33A) provided by InBody.</li></ul> <div><div><p>Power adapter</p></div><div><p>&lt;Plug Type B&gt;</p><p>&lt;Plug Type F&gt;</p></div></div>

Issue	Potential Solution
<ul style="list-style-type: none"> <li>• The height measurement values are incorrect.</li> </ul>	<ul style="list-style-type: none"> <li>• Check if the equipment is installed on a flat and vibration-free floor.</li> <li>• Proper measuring posture must be maintained. Please refer to 'C. Measuring posture' of 'II. Height measurement', for more details on proper measuring posture.</li> <li>• Turn off the power and on again.</li> </ul>
<ul style="list-style-type: none"> <li>• The weight measurement values are incorrect.</li> </ul>	<ul style="list-style-type: none"> <li>• This may occur when warming up is processed abnormally. The zero adjustment will not perform properly, which may lead to inaccurate measurement, when there is an object placed on the footplate. Please remove any objects on the footplate and turn the power off and on again.</li> <li>• This may occur when foreign substance is stuck between the footplate and the footplate screw. In this case, weight measurement will be inaccurate. Please remove the foreign substance before use.</li> <li>• This may occur when the equipment is not leveled. In this case, weight measurement will be inaccurate. Please refer to '2. Setup Menu 9) of 'Setup' and level the equipment by rotating the leveling screws.</li> </ul>

## V. Other

\* The BSM270/BSM270B was manufactured according to the quality control procedure of InBody Co., Ltd. InBody complies with ISO9001, which are international quality management systems.

### A. Safety information

#### WARNING

1. Do not let examinees with physical disability or children to measure by themselves. They may measure with the help of the operator.
2. To avoid electric shock, do not plug or disconnect the power cable with wet hands.
3. Do not play around or jump on the footplate. This may cause serious injuries by falling.
4. To prevent fire or equipment malfunction, always use an outlet designed for the correct power supply.
5. If using a multiple tab, ensure it has a sufficient power capacity or use an extension cable.
6. Do not let the BSM270/270B touch other electronic devices when turned on. This may cause electric shock.
7. Do not disassemble the equipment arbitrarily. This may cause electric shock or product malfunction. There is no parts in the equipment for the users to operate. InBody Co., Ltd. is not responsible for any damage caused by this action.

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## CAUTION

1. **Cleaning**  
Unplug the power cable before cleaning. Wipe the equipment with a soft cloth that is clean and lint-free. Do not spray water or wipe with benzene, thinner, or alcohol. Spraying any liquids may cause malfunction or electric shock.
2. **Peripheral devices**  
Do not let the BSM270/BSM270B touch other electronic devices when turned on. This may cause electric shock.
3. **Installation**  
The equipment should be installed on a flat and vibration-free floor. If the equipment is installed on a surface that is not flat, it may topple during the measurement or the results may be inaccurate.
4. **Use the adapter model provided by the manufacturer.** The use of other adapters can cause product malfunction.
5. Do not let foreign objects like dust and liquid enter the device, as they can cause damage to electrical components.
6. Do not move during the measurement. The measurement values may be inaccurate.
7. Do not bend the head bar with force. The equipment may be damaged.
8. When leveling the equipment, avoid injuring your hands when rotating the leveling screws under the footplate.
9. Dispose of trash according to local laws and regulations.

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## NOTE

1. The measurement values may be inaccurate if the equipment is not installed according to this User's Manual, or due to malfunction caused by interference of peripheral devices.  
Resolve the interference with in the following ways.
  - Separate the power from the peripheral device that is causing interference.
  - Move the equipment to another space.
  - If the problem persists, please contact Customer Support.
2. As excessively high or low temperature, humidity, and pressure may affect the operation of the equipment and cause malfunction, use within the range specified in the product specifications.
3. When transporting BSM270/270B, make sure to use the packing material provided at the time of purchase and avoid applying any strong physical impacts on the equipment during installation or disassembly.
4. Do the use the equipment for any other purpose other than measuring height and weight.
5. Repair and inspection can only be performed by A/S staffs of InBody Co. Ltd. Please contact Product Support for any repairs and inspections.

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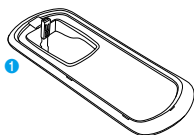
## B. Exterior and Functions

Below is a list of BSM270/BSM270B components and its function.

\* Please check each component of the system for damage prior to installation.

### 1. The head bar

- ① Head bar: A part to measure height.


### 2. Stand

- ① Stand: A part to support the equipment.

### 3. Footplate

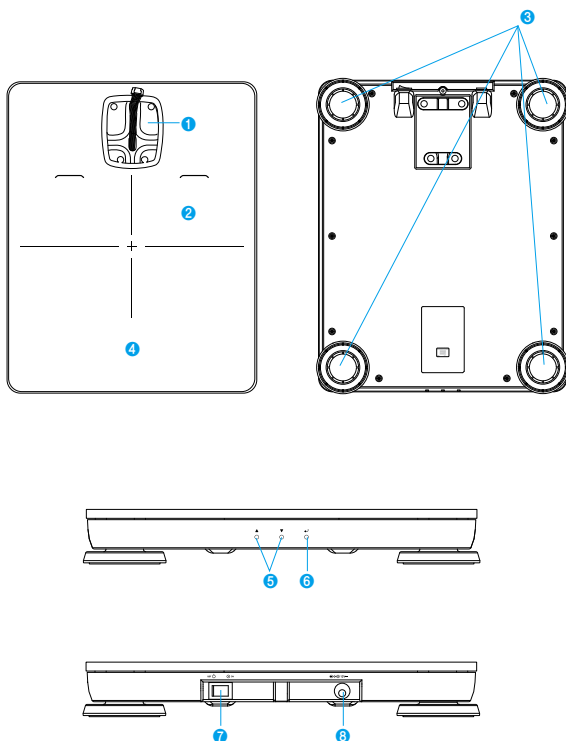- ① Stand groove: A groove on footplate to insert and assemble the stand to the footplate.
- ② Footplate: A plate that examinee can stand on for measurement.
- ③ Footplate adjustment screws: Screws to level the equipment.
- ④ LCD screen: A screen that displays height and weight.
- ⑤ ▲, ▼ Direction buttons: Buttons to adjust settings.
- ⑥ Enter button: A button to select or change settings.
- ⑦ Power switch: A switch to turn the equipment on/off.
- ⑧ Power input terminal: An input for external power source.

## C. Indication symbols

### Safety symbols

WARNING / AVERTISSEMENT, CAUTION / ATTENTION

Note / REMARQUE

### Other symbols

Manufacturer

European Conformity

Serial number

DC power

## D. Product specifications

EN

Product purpose	A device that can automatically measure height and weight	
Measurement items	Height, Weight, BMI	
Measured height	900~2000 mm (2 ft 11.4 inch ~ 6 ft 6.7 inch)	
Height measurement error range	± 1 mm	
Measured weight	10~200 kg (22~440 lb)	
Outcome indication	Measurement outcome is displayed on the LCD screen	
Zero-point adjustment	Zero adjustment is performed during the warmup (3~15 seconds) for weight measurements	
Measurement mode	WH (measures both weight and height) W (measures only weight) H (measures only height)	
Height measurement	AUTO (automatic height measurement after weight measurement)	
	MANUAL (after weight measurement, press the "ENTER" button to measure height)	
Height offset	Can be adjusted between + 50.0 cm and -50.0 cm in 0.1 cm increments	
Weight offset	Can be adjusted between +5.0 kg and -5.0 kg in 0.1 kg increments	
	Can be adjusted between +10.0 lb and -10.0 lb in 0.2 lb increments	
Measurement units	Select between 'kg/cm' and 'lb/in'	
Adapter	Power input	100 - 240 V, 50/60 Hz, 1.2 A
	Power output	DC 12 V, 3.33 A
External interface	Bluetooth * Applies only to the BSM270B unit	
Equipment size	355(W) × 455(L) × 2170(H): mm 139.8(W) × 179.1(L) × 854.3(H): inch	
Equipment weight	14kg(30lb)	
Operating environment	Temperature 10 ~ 40 °C (50 ~ 104 °F), humidity 30 ~ 75 % RH, air pressure 70 ~ 106 kPa	
Transportation and storage environment	Temperature -10 ~ 70 °C (14 ~ 158 °F), humidity 10 ~ 80 % RH, air pressure 50 ~ 106 kPa (No condensation)	

\* Specifications are subject to change without notice for purposes of product improvement.

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TEL: +82-41-581-3003 FAX: +82-41-581-3103

Website: inbody.com E-mail: info@inbody.com

## Representative & Sponsor Information [법인]

### InBody Europe B.V. [NETHERLANDS]

Gyroscoopweg 122, 1042 AZ, Amsterdam, The Netherlands

TEL: +31-20-238-6080 FAX: +31-6-5734-1858

Website: nl.inbody.com E-mail: info.eu@inbody.com

Website (CS): cs.inbody.eu E-mail (CS): support@inbody.com

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Website: de.inbody.com E-mail: erfolg@inbody.com

### InBody UK [UNITED KINGDOM]

11 Phoenix Park, Telford Way, Stephenson Industrial Estate, Coalville LE67 3HB, United Kingdom

TEL: +44-1530-569620

Website: uk.inbody.com E-mail: uk@inbody.com

### InBody Oceania [AUSTRALIA]

Main office: Level 8, 1 York Street, SYDNEY, NSW 2000, Australia

Showroom: U2/82-86 Minnie Street, Southport, Queensland

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WP KL 60000 Malaysia

TEL: +60-3-7732-0790 FAX: +60-3-7733-0790

Website: inbodyasia.com E-mail: info@inbodyasia.com

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Website: inbodymexico.com E-mail: info.mx@inbody.com

### InBody India [INDIA]

57/57 A, 1st Floor, Raj Industrial Complex, Military Road, Marol,

Andheri (East), Mumbai- 400059, Maharashtra, India

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